# Bioclin

## BIOCLIN POCT FIA BNP

REF K363

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#### FINALIDADE

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- 1- WHO. Use of anticoagulants in diagnostic laboratory investigations and stability of blood, plasma and serum samples. WHO/DIL/LAB/99.1 rev. 2, 2002:31.
- 2- Tate, J.; Ward, G. Interferences in immunoassay. Clin. Biochem. Rev., v. 25, n. 2, p. 105-120, 2004.
- 3- Mueller, T. et al. Long-term stability of endogenous B-type natriuretic peptide (BNP) and amino terminal proBNP (NT-ProBNP) in frozen plasma samples. Clin. Chem. Lab. Med., v. 42, n. 8, p. 942-944, 2004.
- 4- Emdin, M. et al. Comparison of brain natriuretic peptide (BNP) and amino-terminal proBNP for early diagnosis of heart failure. J. Clin. Chem., v. 53, p. 1289-1297, 2007.
- 5- De Lemos, J. A.; McGire, D. K.; Drazner, M. H. B-type natriuretic in cardiovascular disease. Lancet, v. 362, p. 316-322, 2003.

6- Pfister, R. et al. The value of natriuretic peptides NT-proBNP and BNP for the assessment of left-ventricular volume and function: A prospective study of 150 patients. Deutsche Medizinische Wochenschrift, v. 127, n. 49, 1946.

7- Ziskoven, D. et al. Calcium Calmodulin antagonists influences the release of Cardiodilatin/ANP from Atrial Cardiocytes". In Kaufmann, W.; Wambach, G. Handbook Endocrinology of the Heart. Berlin: Springer, p. 233-4, 1989.

8- Maisel, A. et al. Rapid measurement of B-type natriuretic peptide in the emergency diagnosis of heart failure. N. Engl. J. Med., v. 347, n. 3, p. 161-167, 2002.

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- WHO. Use of anticoagulants in diagnostic laboratory investigations and stability of blood, plasma and serum samples. WHO/DIL/LAB/99.1 rev. 2, 2002:31.
- Tate, J.; Ward, G. Interferences in immunoassay. Clin. Biochem. Rev., v. 25, n. 2, p. 105-120, 2004.
- Mueller, T. et al. Long-term stability of endogenous B-type natriuretic peptide (BNP) and amino terminal proBNP (NT-ProBNP) in frozen plasma samples. Clin. Chem. Lab. Med., v. 42, n. 8, p. 942-944, 2004.
- Emdin, M. et al. Comparison of brain natriuretic peptide (BNP) and amino-terminal proBNP for early diagnosis of heart failure. J. Clin. Chem., v. 53, p. 1289- 1297, 2007.
- De Lemos, J. A.; McGire, D. K.; Drazner, M. H. B-type

natriuretic in cardiovascular disease. Lancet, v. 362, p. 316-322, 2003.

6- Pfister, R. et al. The value of natriuretic peptides NT-proBNP and BNP for the assessment of left-ventricular volume and function: A prospective study of 150 patients. Deutsche Medizinische Wochenschrift, v. 127, n. 49, 1946.

7- Ziskoven, D. et al. Calcium Calmodulin antagonists influences the release of Cardiodilatin/ANP from Atrial Cardiocytes". In Kaufmann, W.; Wambach, G. Handbook Endocrinology of the Heart. Berlin: Springer, p. 233-4, 1989.

8- Maisel, A. et al. Rapid measurement of B-type natriuretic peptide in the emergency diagnosis of heart failure. N. Engl. J. Med., v. 347, n. 3, p. 161-167, 2002.

9- Bhalla, V.; Willis, S.; Maisel, A.S. B-type natriuretic peptide the level and the drug-partners in the diagnosis of heart failure. Congest Heart Fail, v. 10, n. 1, p. 3-27, 2004.

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### INSTRUCTIONS FOR USE

#### FUNCTION

Test for the quantitative determination of brain natriuretic peptide (BNP) in human biological samples of plasma and whole blood, using a fluorescent immunoassay. For *in vitro* diagnostic use only.

#### PRINCIPLE OF ACTION

**Methodology:** Fluorescent immunoassay.

The BIOCLIN POCT FIA BNP kit is a fluorescent immunoassay based on the “sandwich” detection principle, intended for the quantitative determination of BNP in human plasma or whole blood samples (collected in EDTA). When the sample is added to its application site, an interaction occurs between the BNP molecules present in the sample and the anti-BNP antibodies conjugated to fluorescent microspheres, forming the BNP-antibody conjugate complex. As the sample migrates by capillary action through the nitrocellulose membrane, this complex is captured by the test line, which also consists of anti-BNP antibodies. The sample will continue migrating through the membrane until it reaches the control line, which consists of rabbit anti-IgG antibodies (where rabbit IgG antibodies conjugated to fluorescent microspheres will be captured). At the end of the incubation time, the cassette is read in the immunofluorescence reader. The higher the concentration of BNP present in the sample, the more complexes are immobilized on the test line and, consequently, the greater the fluorescence signal generated and captured by the reader. The concentration of BNP in the sample, in pg/mL, is calculated according to the assay calibration curve (contained in the Identification Card).

#### REAGENTS

**1- Cassette** - Store between 2 and 30 °C. Each sachet contains:

**A)** 01 plastic device with 01 test strip containing:

- Test line region: mouse anti-BNP antibody.

- Control line region: goat anti-rabbit IgG antibody.

- Conjugate: fluorescent microspheres conjugated to mouse anti-BNP antibodies and fluorescent microspheres conjugated to rabbit IgG antibodies.

**B)** 01 silica sachet.

**2- Sample Diluent** - Store between 2 and 30 °C. Contains: 10 mM phosphate-buffered saline (PBS), stabilizers (casein <1%) and preservatives (ProClin™ 300 <0.5%).

**3- Identification Card**

#### PRESENTATION

Reagents	Number of Reagents per Package
<b>1- Cassette</b>	25 units
<b>2- Sample Diluent</b>	25 units (25 x 0.160 mL)
<b>3- Identification Card</b>	1 unit

#### EQUIPMENTS AND OPERATIONAL INPUTS

**Materials in the kit:**

- Reagents described in the table above.

**Required materials not contained in the kit:**

**1-** Pipettes capable of dispensing volumes from 5 to 100 µL with a coefficient of variation lower than 1.5%.

**2-** BIOCLIN POCT FIA Immunofluorescence Reader.

**3-** Stopwatch or clock.

**4-** Quality Control Tools.

#### STORAGE AND TRANSPORTATION CONDITIONS

The storage and transportation temperature should be between 2 °C and 30 °C. Keep away from light and avoid humidity. **Do not freeze.**

#### SPECIAL CARE

**1- For *in vitro* diagnostic use only.**

**2-BIOCLIN POCT FIA BNP should only be used in conjunction with the BIOCLIN POCT FIA Immunofluorescence Reader.**

**3-** Strictly follow the proposed methodology to obtain accurate results.

**4-** The cassettes (test devices) do not contain contaminating substances, but since the samples used are biological, safety routines must be strictly followed and the use of disposable gloves is essential.

**5-** Open the aluminum envelope to remove the cassette only at the time of use.

**6-** The kit should not be frozen or exposed to high temperatures (above 45 °C), as this will cause deterioration.

**7-** The reagents, as well as the samples, must reach room temperature before starting the test.

**8-** Do not mix reagents from kits with the same batch or with different batch numbers.

**9-** Do not use the kit after its expiration date. The expiration date is indicated on the product label.

**10-** The test device (R1- Cassette) and sample dilution tubes (R2- Sample Diluent) are disposable and cannot be reused.

**11-** The kit materials that have come into contact with the patient's sample must be disposed of in a special way. Pipettes, cassettes (test devices) and diluent vials must be disposed of in biological waste.

**12-** Secondary packaging (box), aluminum envelope and silica sachet can be disposed of in regular waste.

**13-** We recommend applying local, state and federal environmental protection standards so that the disposal of reagents and biological material is done in accordance with current legislation.

**14-** To obtain information related to biosafety or in case of accidents with the product, consult the SDS (Safety Data Sheet) available on the website [www.bioclin.com.br](http://www.bioclin.com.br) or by

requesting it through Quibasa's SAC (Customer Advisory Service).

**15-** Do not use the product if the packaging is damaged.

**16-** It is essential that the instruments and equipment used are properly calibrated and undergo periodic maintenance.

#### SAMPLES

**Plasma (EDTA)**

Samples should be collected by venipuncture in tubes containing EDTA as an anticoagulant. To avoid hemolysis, centrifuge the samples to obtain plasma and test them as soon as possible. Do not keep samples at room temperature for extended periods. Plasma samples can be stored under refrigeration between 2 and 8 °C for a maximum of 8 hours. If samples cannot be analyzed within 8 hours, they can be stored for up to 30 days at -20 °C. Plasma samples stored for longer than recommended and samples collected with anticoagulants other than those mentioned should not be used. Highly hemolyzed, lipemic samples and those containing precipitates should be avoided, as they may provide inconsistent results.

**Whole Blood (EDTA)**

Whole blood samples should be collected by venipuncture in tubes containing EDTA as an anticoagulant. Samples should be tested immediately or as soon as possible after collection. Whole blood collected in tubes containing anticoagulant can be stored under refrigeration, between 2 and 8 °C, for a maximum period of 8 hours. Do not use samples collected with anticoagulants other than those mentioned and do not freeze whole blood samples.

#### PROCESS DESCRIPTION

**Stability After Opening**

The stability test results show that the BIOCLIN POCT FIA BNP kit is stable after opening for up to 70 minutes. This stability may vary depending on the conditions of use of the product and the environment. However, for safety reasons, we recommend that the product should be used immediately after removing the cassette from the sachet.

#### TECHNIQUE

**Check the BIOCLIN POCT FIA Immunofluorescence Reader manual for complete instructions on handling the equipment.**

Before starting the test, allow all reagents (Cassette and Sample Diluent) and samples to stabilize at room temperature (15-30 °C) for at least 40 minutes.

**1-** Check if the information contained in the Identification Card has been properly entered into the equipment (with each new batch of the kit, the calibration and validity data contained in the Identification Card must be transferred to the equipment).

**2-** Remove the cassette (test device) from its protective packaging, place it on a clean, level surface and identify it appropriately.

**3-** Make sure that the Sample Diluent solution (Reagent No. 2) is at the bottom of the dilution tube.

**4-** Add 50 µL of sample to a dilution tube containing the Sample Diluent.

**5-** Close the tube cap and mix by inverting the sample 6 to 8 times until the sample and diluent are completely homogenized.

TO OBTAIN THE INSTRUCTIONS FOR USE IN PRINTED FORMAT, AT NO ADDITIONAL COST, CONTACT CUSTOMER ADVISORY SERVICE:

SAC: +55 (31) 3439 5454 / 0800 031 5454 / [sac@bioclin.com.br](mailto:sac@bioclin.com.br)

**6-** Pipette 50 µL of diluted and properly mixed sample into the sample application hole of the cassette.

**7- Option 1:** Insert the cassette into the equipment immediately after applying the sample and select the “**Timing Test**” option. After 15 minutes, the equipment will take the reading and the result will appear on the display.

**Option 2:** After adding the sample, wait 15 minutes. After the time has elapsed, insert the cassette into the equipment and select the “**Quick Test**” option. The equipment will take the reading immediately and the result will appear on the display.

#### RESULTS

The BIOCLIN POCT FIA Immunofluorescence Reader automatically calculates the BNP concentration using the calibration curve contained in the R3 (Identification Card). The information from the Identification Card must be properly entered into the equipment with each new batch of reagents.

Quantitative results are measured in pg/mL. The measurement range of the kit is between 10 and 5,000 pg/mL (quantification limit and maximum point of the curve, respectively). Results below the lower detection limit are expressed as < 10 pg/mL. Results above the upper detection limit are expressed as > 5,000 pg/mL.

#### PROCESS LIMITATIONS

**1-** It is important to use the correct sample volume, as lower or higher volumes may produce erroneous results.

**2-** The reaction reading time must be followed according to the established technique, in order to avoid false interpretations of the results.

**3-** Reactive results must be confirmed with another test and available clinical information. The isolated test results cannot be used for definitive diagnosis.

**4-** The results provided by this kit must be interpreted by the responsible medical professional, and are not the sole criterion for determining the diagnosis and/or treatment of the patient.

#### REFERENCE VALUES

Normal	<100 pg/mL
--------	------------

**Conversion:** 1 pg/mL = 0.289 pmol/L

These values should be used as a guide, and each laboratory should create its own range of reference values according to the population served. The results provided by this kit should be interpreted by the responsible medical professional, and are not the sole criterion for determining the diagnosis and/or treatment of the patient.

#### INTERFERENTS

No interference was observed by bilirubin up to 350 µmol/L, triglycerides up to 40 mmol/L, hemoglobin up to 2 g/L, rheumatoid factor up to 1,500 IU/mL, heterophile antibodies up to 1,000 ng/mL, and antinuclear antibodies up to 1:640.

#### CROSS-REACTIVITY

No false-positive results were observed for adrenomedullin up to 1.0 ng/mL, aldosterone up to 1.0 ng/mL, angiotensin I up to 1.0 ng/mL, angiotensin II up to 1.0 ng/mL, angiotensin III up to 1.0 ng/mL, atrial natriuretic peptide up to 3.0 µg/mL, brain natriuretic peptide up to 1.0 µg/mL, C-type

natriuretic peptide up to 2.0 µg/mL, renin up to 50.0 ng/mL, and urodilatin up to 3.0 µg/mL. Despite the results found, the possibility of cross-reactivity with other analytes or with higher concentrations cannot be completely ruled out. The final diagnosis should consider the patient's clinical data together with other laboratory data.

#### INTERNAL QUALITY CONTROL

The Clinical Laboratory must have an internal quality control program, where procedures, standards, limits and tolerance for variations are clearly established. It is important to emphasize that all measurement systems present a characteristic analytical variability, which must be monitored by the laboratories themselves. For this purpose, it is recommended to use controls, which allow the assessment of the precision and accuracy of the dosages.

#### PRODUCT PERFORMANCE

##### ACCURACY

###### Comparison of Methods

The BIOCLIN POCT FIA BNP kit was compared with another commercially available BNP assay method. 112 analyses were performed, and the results were evaluated. The linear equation obtained was  $Y = 0.9769x + 21.072$  and the correlation coefficient was 0.9688. Based on these results, it can be concluded that the kit presents good methodological specificity.

##### PRECISION

###### Repeatability

Repeatability was calculated from 10 successive determinations, using 2 samples with different values, obtaining the following concentration results:

REPEATABILITY	SAMPRE	
	1	2
Mean	204.95	974.20
Standart Deviation	14.36	65.79
Coefficient of Variation	7.0	6.8

###### Reproducibility

Reproducibility was calculated from 10 successive determinations over 3 consecutive days, using 2 samples with different values, obtaining the following concentration results:

REPRODUCIBILITY	SAMPRE	
	1	2
Mean	200.10	1006.60
Standart Deviation	16.3	74.9
Coefficient of Variation	8.1	7.4

##### ANALYTICAL SENSITIVITY

The Limit of Blank (LoB) of the BIOCLIN POCT FIA BNP product is 2.5 pg/mL, the Limit of Detection (LoD) is 5.0 pg/mL and the Limit of Quantification (LoQ) is 10.0 pg/mL.

##### LINEARITY

The BIOCLIN POCT FIA BNP kit is linear up to 5,000 pg/mL (highest point of the calibration curve).

##### HIGH DOSE PROZONE EFFECT

No high-dose prozone effect was observed at up to 60,000 pg/mL of BNP.

#### DIAGNOSTIC SIGNIFICANCE

Brain natriuretic peptide (BNP) is one of the natriuretic peptides involved in the control of cardiovascular function, along with atrial natriuretic peptide (ANP) and C-type natriuretic peptide (CNP). BNP is the physiologically active hormone, generated after the cleavage of the N-terminal portion of the prohormone proBNP. This prohormone is produced by the ventricles through stimulation by the stretching of cardiomyocytes, and to a lesser extent by the brain, kidney, aorta, and adrenal gland. Like ANP, BNP binds to NPRA receptors and activates cellular signals that maintain the body's electrolyte and fluid balance, regulating blood pressure. Despite having a 10 times lower affinity for its receptors than ANP, BNP has a half-life of 20 minutes in the bloodstream, significantly longer than the 3 minutes of ANP. NT-proBNP has the longest half-life among them, approximately 120 minutes. Therefore, BNP, like NT-proBNP, is an important marker for the diagnosis and prognosis of heart failure (HF) and left ventricular dysfunction, useful for distinguishing HF from other causes in symptomatic patients presenting with recent-onset dyspnea. Heart failure is a syndrome characterized by inadequate body perfusion due to a structural or functional cardiac abnormality, rendering the body unable to meet its metabolic demands. HF is a progressive disease where most deaths result from its worsening, leading to sudden death. Due to its shorter half-life, BNP accurately reflects the current ventricular status, making it useful in monitoring acute heart failure.

#### BIBLIOGRAPHIC REFERENCES

- 1- WHO. Use of anticoagulants in diagnostic laboratory investigations and stability of blood, plasma and serum samples. WHO/DIL/LAB/99.1 rev. 2, 2002:31.
- 2- Tate, J.; Ward, G. Interferences in immunoassay. Clin. Biochem. Rev., v. 25, n. 2, p. 105-120, 2004.
- 3- Mueller, T. et al. Long-term stability of endogenous B-type natriuretic peptide (BNP) and amino terminal proBNP (NT-ProBNP) in frozen plasma samples. Clin. Chem. Lab. Med., v. 42, n. 8, p. 942-944, 2004.
- 4- Emdin, M. et al. Comparison of brain natriuretic peptide (BNP) and amino-terminal proBNP for early diagnosis of heart failure. J. Clin. Chem., v. 53, p. 1289- 1297, 2007.
- 5- De Lemos, J. A.; McGire, D. K.; Drazner, M. H. B-type natriuretic in cardiovascular disease. Lancet, v. 362, p. 316-322, 2003.
- 6- Pfister, R. et al. The value of natriuretic peptides NT-proBNP and BNP for the assessment of left-ventricular volume and function: A prospective study of 150 patients. Deutsche Medizinische Wochenschrift, v. 127, n. 49, 1946.
- 7- Ziskoven, D. et al. Calcium Calmodulin antagonists influences the release of Cardiodilatin/ANP from Atrial Cardiocytes". In Kaufmann, W.; Wambach, G. Handbook Endocrinology of the Heart. Berlin: Springer, p. 233-4, 1989.
- 8- Maisel, A. et al. Rapid measurement of B-type natriuretic peptide in the emergency diagnosis of heart failure. N. Engl. J. Med., v. 347, n. 3, p. 161-167, 2002.
- 9- Bhalla, V.; Willis, S.; Maisel, A.S. B-type natriuretic peptide the level and the drug-partners in the diagnosis of heart failure. Congest Heart Fail, v. 10, n. 1, p. 3-27, 2004.
- 10- Bioclin – Archive Files.

#### QUALITY ASSURANCE

Before being released for consumption, all Bioclin reagents are tested by the Quality Control Department. The quality of the reagents is assured until the expiration date mentioned on the presentation packaging, provided they are stored and transported under appropriate conditions.

#### QUIBASA QUÍMICA BÁSICA Ltda

Rua Teles de Menezes, 92 - Santa Branca  
CEP 31565-130 - Belo Horizonte - MG - Brasil  
Phone: +55 (31) 3439.5454 | E-mail: bioclin@bioclin.com.br  
CNPJ: 19.400.787/0001-07 - Made in Brazil

#### CUSTOMER SERVICE

Customer Advisory Service  
Phone.: 0800 0315454 | E-mail: sac@bioclin.com.br

**ANVISA registration number for BIOCLIN POCT FIA BNP kit:** 10269360482

**Review:** December/2025

#### UNIVERSAL SYMBOLOGY

 REF	CATALOG NUMBER	 MADE BY
 LOT	LOT NUMBER	 CONTROL
 MANUFACTURING DATE		 POSITIVE CONTROL
 VALIDITY DATE (last day of the month)		 NEGATIVE CONTROL
 TEMPERATURE LIMIT (store)		 BIOLOGICAL RISK
 CONTENT IS SUFFICIENT FOR <N> TEST		 FLAMMABLE
 SEE INSTRUCTIONS FOR USE		 CORROSIVE
 IVD IN VITRO DIAGNOSTIC PRODUCT		 TOXIC
 KEEP AWAY FROM SUNLIGHT		 DO NOT USE IF PACKAGE IS DAMAGED
 DO NOT REUSE		 PRODUCT STERILIZED
 CAUTION		 DANGER