Bioclin

BIOCLIN POCT FIA NT-PROBNP

REF K340

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FINALIDADE

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- 1- WHO. Use of anticoagulants in diagnostic laboratory investigations and stability of blood, plasma and serum samples. WHO/DIL/LAB/99.1 rev. 2, 2002:31.
- 2- Tate, J.; Ward, G. Interferences in immunoassay. Clin. Biochem. Rev., v. 25, n. 2, p. 105-120, 2004.
- 3- Mueller, T. et al. Long-term stability of endogenous B-type natriuretic peptide (BNP) and amino terminal proBNP (NT-ProBNP) in frozen plasma samples. Clin. Chem. Lab. Med., v. 42, n. 8, p. 942-944, 2004.
- 4- Emdin, M. et al. Comparison of brain natriuretic peptide (BNP) and amino-terminal proBNP for early diagnosis of heart failure. J. Clin. Chem., v. 53, p. 1289-1297, 2007.
- 5- Januzzi, J. L. et al. The N-terminal pro-BNP investigation

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- WHO. Use of anticoagulants in diagnostic laboratory investigations and stability of blood, plasma and serum samples. WHO/DIL/LAB/99.1 rev. 2, 2002:31.
- Tate, J.; Ward, G. Interferences in immunoassay. Clin. Biochem. Rev., v. 25, n. 2, p. 105-120, 2004.
- Mueller, T. et al. Long-term stability of endogenous B-type natriuretic peptide (BNP) and amino terminal proBNP (NT-ProBNP) in frozen plasma samples. Clin. Chem. Lab. Med., v. 42, n. 8, p. 942-944, 2004.
- Emdin, M. et al. Comparison of brain natriuretic peptide (BNP) and amino-terminal proBNP for early diagnosis of heart failure. J. Clin. Chem., v. 53, p. 1289-1297, 2007.
- Januzzi, J. L. et al. The N-terminal pro-BNP investigation

of dyspnea in the emergency department (PRIDE) study. J. Am. J. Cardiol., v.95, p. 948-954, 2005.

6- De Lemos, J. A.; McGire, D. K.; Drazner, M. H. B-type natriuretic in cardiovascular disease. Lancet, v. 362, p. 316-322, 2003.

7- Pfister, R. et al. The value of natriuretic peptides NT-proBNP and BNP for the assessment of left-ventricular volume and function: A prospective study of 150 patients. Deutsche Medizinische Wochenschrift, v. 127, n. 49, 1946.

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INSTRUCTIONS FOR USE

FUNCTION

Test for the quantitative determination of the N-terminal portion of the prohormone brain natriuretic peptide (NT-proBNP) in human biological samples of serum, plasma, and whole blood, through fluorescent immunoassay. For *in vitro* diagnostic use only.

PRINCIPLE OF ACTION

Methodology: Fluorescent immunoassay.

The BIOCLIN POCT FIA NT-ProBNP kit is a fluorescent immunoassay based on the sandwich detection principle, designed for the quantitative determination of the N-terminal portion of the prohormone brain natriuretic peptide (NT-proBNP) in human serum, plasma, or whole blood samples (collected in EDTA or heparin). When the sample is added to its application site, the NT-proBNP molecules present in the sample interact with the anti-NT-proBNP antibodies conjugated to fluorescent microspheres, forming the NT-proBNP-antibody conjugate complex. As the sample migrates by capillary action across the nitrocellulose membrane, this complex is captured by the test line, which also consists of anti-NT-proBNP antibodies. The sample will continue to migrate across the membrane until it reaches the control line, which consists of anti-rabbit IgG antibodies (where rabbit IgG antibodies conjugated to fluorescent microspheres will be captured). At the end of the incubation period, the cassette is read on the immunofluorescence reader. The higher the concentration of NT-proBNP present in the sample, the more complexes are immobilized on the test line and, consequently, the higher the fluorescence signal generated and captured by the reader. The concentration of NT-proBNP in the sample, in pg/mL, is calculated according to the assay calibration curve (contained on the Identification Card).

REAGENTS

1- Cassette - Store between 2 and 30 °C. Each sachet contains:

A) 01 plastic device with 01 test strip containing:

- Test line region: mouse anti-NT-proBNP antibody.

- Control line region: goat anti-rabbit IgG antibody.

- Conjugate: fluorescent microspheres conjugated to mouse anti-NT-proBNP antibodies and fluorescent microspheres conjugated to rabbit IgG antibodies.

B) 01 silica sachet.

2- Sample Diluent - Store between 2 and 30 °C. Contains: 10 mM phosphate-buffered saline (PBS), stabilizers (casein <1%) and preservatives (ProClin™ 300 <0.5%).

3-Identification Card

PRESENTATION

REAGENTS	NUMBER OF REAGENTS PER PACKAGE
1- Cassette	25 units
2- Sample Diluent	25 units (25 x 0.160 mL)
3- Identification Card	1 unit

EQUIPMENTS AND OPERATIONAL INPUTS

Materials in the kit:

- Reagents described in the table above.

Required materials not contained in the kit:

1- Pipettes capable of dispensing volumes from 5 to 100 µL with a coefficient of variation lower than 1.5%.

2- BIOCLIN POCT FIA Immunofluorescence Reader.

3- Stopwatch or clock.

4- Quality Control Tools.

STORAGE AND TRANSPORTATION CONDITIONS

The storage and transportation temperature should be between 2 °C and 30 °C. Keep away from light and avoid humidity. **Do not freeze.**

SPECIAL CARE

1- For *in vitro* diagnostic use only.

2-BIOCLIN POCT FIA NT-PROBNP should only be used in conjunction with the BIOCLIN POCT FIA Immunofluorescence Reader.

3- Strictly follow the proposed methodology to obtain accurate results.

4- The cassettes (test devices) do not contain contaminating substances, but since the samples used are biological, safety routines must be strictly followed and the use of disposable gloves is essential.

5- Open the aluminum envelope to remove the cassette only at the time of use.

6- The kit should not be frozen or exposed to high temperatures (above 45 °C), as this will cause deterioration.

7- The reagents, as well as the samples, must reach room temperature before starting the test.

8- Do not mix reagents from kits with the same batch or with different batch numbers.

9- Do not use the kit after its expiration date. The expiration date is indicated on the product label.

10- The test device (R1- Cassette) and sample dilution tubes (R2- Sample Diluent) are disposable and cannot be reused.

11- The kit materials that have come into contact with the patient's sample must be disposed of in a special way. Pipettes, cassettes (test devices) and diluent vials must be disposed of in biological waste.

12- Secondary packaging (box), aluminum envelope and silica sachet can be disposed of in regular waste.

13- We recommend applying local, state and federal environmental protection standards so that the disposal of reagents and biological material is done in accordance with current legislation.

14- To obtain information related to biosafety or in case of accidents with the product, consult the SDS (Safety Data Sheet) available on the website www.bioclin.com.br or by

requesting it through Quibasa's SAC (Customer Advisory Service).

15- Do not use the product if the packaging is damaged.

16- It is essential that the instruments and equipment used are properly calibrated and undergo periodic maintenance.

SAMPLES

Serum or Plasma (EDTA or Heparin)

Samples should be collected by venipuncture, in tubes with or without anticoagulant (EDTA or Heparin). To avoid hemolysis, centrifuge the samples to obtain serum/plasma and test them as quickly as possible. Do not keep samples at room temperature for long periods. Serum and plasma samples can be stored refrigerated, between 2 and 8°C, for a maximum of 3 days. If samples cannot be analyzed within 3 days, they can be stored for up to 30 days at -20°C. Serum and plasma samples stored for longer than the recommended periods and samples collected with anticoagulants other than those mentioned should not be used. Highly hemolyzed, lipemic, and precipitate-containing samples should be avoided, as they may provide inconsistent results.

Whole Blood (EDTA or Heparin)

Whole blood samples should be collected by venipuncture in tubes containing heparin or EDTA as an anticoagulant. Samples should be tested immediately or as soon as possible after collection. Whole blood collected in tubes containing anticoagulants can be stored refrigerated at 2–8°C for a maximum of 2 days. Do not use samples collected with anticoagulants other than those mentioned, and do not freeze whole blood samples.

PROCESS DESCRIPTION

Stability After Opening

The stability test results show that the BIOCLIN POCT FIA NT-PROBNP kit is stable after opening for up to 70 minutes. This stability may vary depending on the conditions of use of the product and the environment. However, for safety reasons, we recommend that the product should be used immediately after removing the cassette from the sachet.

TECHNIQUE

Check the BIOCLIN POCT FIA Immunofluorescence Reader manual for complete instructions on handling the equipment.

Before starting the test, allow all reagents (Cassette and Sample Diluent) and samples to stabilize at room temperature (15-30 °C) for at least 40 minutes.

1- Check if the information contained in the Identification Card has been properly entered into the equipment (with each new batch of the kit, the calibration and validity data contained in the Identification Card must be transferred to the equipment).

2- Remove the cassette (test device) from its protective packaging, place it on a clean, level surface and identify it appropriately.

3- Make sure that the Sample Diluent solution (Reagent No. 2) is at the bottom of the dilution tube.

4- Add 50 µL of sample to a dilution tube containing the Sample Diluent.

5- Close the tube cap and mix by inverting the sample 6 to 8 times until the sample and diluent are completely homogenized.

TO OBTAIN THE INSTRUCTIONS FOR USE IN PRINTED FORMAT, AT NO ADDITIONAL COST, CONTACT CUSTOMER ADVISORY SERVICE:

SAC: +55 (31) 3439 5454 / 0800 031 5454 / sac@bioclin.com.br

6- Pipette 50 µL of diluted and properly mixed sample into the sample application hole of the cassette.

7- Option 1: Insert the cassette into the equipment immediately after applying the sample and select the “**Timing Test**” option. After 15 minutes, the equipment will take the reading and the result will appear on the display.

Option 2: After adding the sample, wait 15 minutes. After the time has elapsed, insert the cassette into the equipment and select the “**Quick Test**” option. The equipment will take the reading immediately and the result will appear on the display.

RESULTS

The BIOCLIN POCT FIA Immunofluorescence Reader automatically calculates the NT-proBNP concentration using the calibration curve contained in the R3 (Identification Card). The information on the Identification Card must be properly entered into the equipment for each new batch of reagents.

Quantitative results are measured in pg/mL. The kit's measurement range is between 150 and 35,000 pg/mL (limit of quantification and maximum point of the curve, respectively). Results below the lower limit of detection are expressed as <150 pg/mL. Results above the upper limit of detection are expressed as >35,000 pg/mL.

PROCESS LIMITATIONS

1- It is important to use the correct sample volume, as lower or higher volumes may produce erroneous results.

2- The reaction reading time must be followed according to the established technique, in order to avoid false interpretations of the results.

3- Reactive results must be confirmed with another test and available clinical information. The isolated test results cannot be used for definitive diagnosis.

4- The results provided by this kit must be interpreted by the responsible medical professional, and are not the sole criterion for determining the diagnosis and/or treatment of the patient

REFERENCE VALUES

<75 years	<300 pg/mL
≥75 years	<450 pg/mL

These values should be used as a guide, and each laboratory should create its own range of reference values according to the population served. The results provided by this kit should be interpreted by the responsible medical professional, and are not the sole criterion for determining the diagnosis and/or treatment of the patient.

INTERFERENTS

No interference was observed by bilirubin up to 350 µmol/L, triglycerides up to 40 mmol/L, hemoglobin up to 2 g/L, rheumatoid factor up to 1,500 IU/mL, heterophile antibodies up to 1,000 ng/mL, and antinuclear antibodies up to 1:640.

CROSS-REACTIVITY

No false-positive results were observed for adrenomedullin up to 1.0 ng/mL, aldosterone up to 1.0 ng/mL, angiotensin I up to 1.0 ng/mL, angiotensin II up to 1.0 ng/mL, angiotensin

III up to 1.0 ng/mL, atrial natriuretic peptide up to 3.0 µg/mL, brain natriuretic peptide up to 1.0 µg/mL, C-type natriuretic peptide up to 2.0 µg/mL, renin up to 50.0 ng/mL, the possibility of cross-reactivity with other analytes or with higher concentrations cannot be completely ruled out. The final diagnosis should consider the patient's clinical data together with other laboratory data

INTERNAL QUALITY CONTROL

The Clinical Laboratory must have an internal quality control program, where procedures, standards, limits and tolerance for variations are clearly established. It is important to emphasize that all measurement systems present a characteristic analytical variability, which must be monitored by the laboratories themselves. For this purpose, it is recommended to use controls, which allow the assessment of the precision and accuracy of the dosages.

PRODUCT PERFORMANCE

ACCURACY

Comparison of Methods

The BIOCLIN POCT FIA NT-ProBNP kit was compared with another commercially available NT-proBNP assay method. 110 analyses were performed, and the results were evaluated. The linear equation obtained was $Y = 0.979x + 111.99$ and the correlation coefficient was 0.9727. Based on these results, it can be concluded that the kit presents good methodological specificity.

PRECISION

Repeatability

Repeatability was calculated from 10 successive determinations, using 3 samples with different values, obtaining the following concentration results

REPEATABILITY	SAMPRE		
	1	2	3
Mean	259	2499	24806
Standart Deviation	18.53	144.73	1866.66
Coefficient of Variation	7.1	5.8	7.5

Reproducibility

Reproducibility was calculated from 10 successive determinations over 3 consecutive days, using 2 samples with different values, obtaining the following concentration results:

REPRODUCIBILITY	SAMPRE		
	1	2	3
Mean	251	2541	24737
Standart Deviation	17.8	178.3	1724.1
Coefficient of Variation	7.1	7.0	7.0

ANALYTICAL SENSITIVITY

The Limit of Blank (LoB) of the BIOCLIN POCT FIA NT-ProBNP product is 50 pg/mL, the Limit of Detection (LoD) is 100 pg/mL and the Limit of Quantification (LoQ) is 150 pg/mL.

LINEARITY

The BIOCLIN POCT FIA NT-PROBNP kit is linear up to 35.000 pg/mL (highest point of the calibration curve).

HIGH DOSE PROZONE EFFECT

No high-dose prozone effect was observed at up to 115.000 pg/mL of NT-proBNP.

DIAGNOSTIC SIGNIFICANCE

Brain natriuretic peptide (BNP) is one of the natriuretic peptides important for controlling cardiovascular function (which also includes atrial natriuretic peptide and C-type natriuretic peptide), and is crucial for the body's electrolyte and fluid balance. The N-terminal portion of BNP is a cleavage product of the prohormone proBNP, produced by the ventricles through stimulation by cardiomyocyte stretching, which also generates the physiologically active hormone BNP. NT-proBNP is also secreted by other organs, to a lesser extent, such as the brain, kidney, aorta, and adrenal gland. NT-proBNP is an important marker for the diagnosis and monitoring of heart failure (HF), as ventricular dysfunction is one of the main precursors of HF. NT-proBNP is a better marker than BNP itself because it is more stable in the bloodstream and has a longer half-life of approximately 60 to 120 minutes. Heart failure is a syndrome characterized by inadequate body perfusion due to a structural or functional cardiac abnormality, rendering the body unable to meet its metabolic demands. Heart failure is a progressive disease in which most deaths result from its worsening, leading to sudden death. NT-ProBNP levels can be used both in the initial diagnosis and in monitoring the progression of heart failure. Furthermore, its measurement can also be used to identify patients at risk of cardiotoxicity due to the use of certain types of antitumor drugs or interventions that cause fluid retention.

BIBLIOGRAPHIC REFERENCES

- 1- WHO. Use of anticoagulants in diagnostic laboratory investigations and stability of blood, plasma and serum samples. WHO/DIL/LAB/99.1 rev. 2, 2002:31.
- 2- Tate, J.; Ward, G. Interferences in immunoassay. Clin. Biochem. Rev., v. 25, n. 2, p. 105-120, 2004.
- 3- Mueller, T. et al. Long-term stability of endogenous B-type natriuretic peptide (BNP) and amino terminal proBNP (NT-ProBNP) in frozen plasma samples. Clin. Chem. Lab. Med., v. 42, n. 8, p. 942-944, 2004.
- 4- Emdin, M. et al. Comparison of brain natriuretic peptide (BNP) and amino-terminal proBNP for early diagnosis of heart failure. J. Clin. Chem., v. 53, p. 1289- 1297, 2007.
- 5- Januzzi, J. L. et al. The N-terminal pro-BNP investigation of dyspnea in the emergency department (PRIDE) study. J. Am. J. Cardiol., v.95, p. 948-954, 2005.
- 6- De Lemos, J. A.; McGire, D. K.; Drazner, M. H. B-type natriuretic in cardiovascular disease. Lancet, v. 362, p. 316-322, 2003.
- 7- Pfister, R. et al. The value of natriuretic peptides NT-proBNP and BNP for the assessment of left-ventricular volume and function: A prospective study of 150 patients. Deutsche Medizinische Wochenschrift, v. 127, n. 49, 1946.
- 8- Bioclin – Archive Data.

QUALITY ASSURANCE

Before being released for consumption, all Bioclin reagents are tested by the Quality Control Department. The quality of the reagents is assured until the expiration date mentioned on the presentation packaging, provided they are stored and transported under appropriate conditions.

QUIBASA QUÍMICA BÁSICA Ltda

Rua Teles de Menezes, 92 - Santa Branca
CEP 31565-130 - Belo Horizonte - MG - Brasil
Phone: +55 (31) 3439.5454 | E-mail: bioclin@bioclin.com.br
CNPJ: 19.400.787/0001-07 - Made in Brazil

CUSTOMER SERVICE

Customer Advisory Service
Phone.: 0800 0315454 | E-mail: sac@bioclin.com.br

ANVISA registration number for BIOCLIN POCT FIA NT-PROBNP kit: 10269360478

Review: September/2025

UNIVERSAL SYMBOLOGY

	CATALOG NUMBER		MADE BY
	LOT NUMBER		CONTROL
	MANUFACTURING DATE		POSITIVE CONTROL
	VALIDITY DATE (last day of the month)		NEGATIVE CONTROL
	TEMPERATURE LIMIT (store)		BIOLOGICAL RISK
	CONTENT IS SUFFICIENT FOR <N> TEST		FLAMMABLE
	SEE INSTRUCTIONS FOR USE		CORROSIVE
	IN VITRO DIAGNOSTIC PRODUCT		TOXIC
	KEEP AWAY FROM SUNLIGHT		DO NOT USE IF PACKAGE IS DAMAGED
	DO NOT REUSE		PRODUCT STERILIZED
	CAUTION		DANGER