BIOÍONS BUFFER

REF B029

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BIOIONS BUFFER

TO OBTAIN THE INSTRUCTIONS FOR USE IN PRINTED FORMAT, AT NO ADDITIONAL COST,
CONTACT CUSTOMER ADVISORY SERVICE:

REF B029

SAC: +55 (31) 3439 5454 / 0800 031 5454 / sac@bioclin.com.br

INSTRUCTIONS FOR USE

FUNCTION

Reagent used to dilute samples before being dosed in the flow cells of the electrodes. Only for *in vitro* diagnostic use.

Indicated for:

Automatic biochemistry analyzers with ISE module.

REAGENTS

Number 1 - Buffer - Store between 2 to 30°C. Contains: Buffer solution containing Formaldehyde <1% w/v, Phosphoric Acid <1% w/v and conditioning agents.

PRESENTATION

Presentation	Reagent n° 1
1	2 L

TRANSPORTATION AND STORAGE CONDITIONS

The storage and transport temperature should be 2 to 30°C. Keep out of the light and avoid moisture. **Do not freeze.**

SPECIAL CARE

- 1- For *in vitro* diagnostic use only.**
- 2-** Strictly follow the methodology proposed to obtain exact results.
- 3-** Water used in material cleaning must to be recent and free of contaminants.
- 4-** Do not use the product in case of change in appearance or turbidity.
- 5-** We recommend applying the local, state and federal rules for environmental protection, so that disposal of reagents and biological material can be made in accordance with current legislation.
- 6-** To obtain information related to biosafety or in case of accidents with the product, consult the SDS (Safety Data Sheet) available on the website www.bioclin.com.br or upon request by the SAC (Advisory Service Customer) of Quibasa.
- 7-** Do not use the product in case of damaged packaging.
- 8-** It is essential that the instruments and equipments used are properly calibrated and subjected to periodic maintenance.

PROCESS DESCRIPTION

The use of this solution should be according to the manufacturer's manual.

For more information, contact SAC.

Attention: After opening, the Bioions Buffer is stable for 8 weeks when installed in the equipment.

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QUALITY ASSURANCE

Before being released for consumption, all **Bioclin** reagents are tested by the Department of Quality Control. The quality of reagents is assured until expiration date stated on the presentation packaging, when stored and transported under appropriate conditions.

QUIBASA QUÍMICA BÁSICA Ltda

Rua Teles de Menezes, 92 - Santa Branca
CEP 31565-130 - Belo Horizonte - MG - Brasil
Tel.: +55 31 3439.5454 | E-mail: bioclin@bioclin.com.br
CNPJ: 19.400.787/0001-07 – Made in Brazil

CUSTOMER SERVICE

Customer Advisory Service
Phone.: 0800 0315454 | E-mail: sac@bioclin.com.br

ANVISA registration for Bioions Buffer kit: 10269360320

Review: August/2024

UNIVERSAL SYMBOLOGY

	CATALOG NUMBER		MADE BY
	LOT NUMBER		CONTROL
	MANUFACTURING DATE		POSITIVE CONTROL
	VALIDITY DATE (last day of the month)		NEGATIVE CONTROL
	TEMPERATURE LIMIT (store)		BIOLOGICAL RISK
	CONTENT IS SUFFICIENT FOR <N> TEST		FLAMMABLE
	SEE INSTRUCTIONS FOR USE		CORROSIVE
	IN VITRO DIAGNOSTIC PRODUCT		TOXIC
	KEEP AWAY FROM SUNLIGHT		DO NOT USE IF PACKAGE IS DAMAGED
	DO NOT REUSE		PRODUCT STERILIZED
	CAUTION		DANGER