BIOÍONS CAL ISE UR

REF B025

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REF **B025**

INSTRUCTIONS FOR USE

TO OBTAIN THE INSTRUCTIONS FOR USE IN PRINTED FORMAT, AT NO ADDITIONAL COST, CONTACT CUSTOMER ADVISORY SERVICE:

SAC: +55 (31) 3439 5454 / 0800 031 5454 / sac@bioclin.com.br

FUNCTION

Reagent used as calibrator in ISE modules coupled to various automatic biochemical analyzers for electrolyte (Na^+ , K^+ , Cl^-) dosing in urine samples. For *in vitro* diagnostic use only.

Indicated for:

Automatic biochemistry analyzers with ISE module.

REAGENTS

Number 1 - Ions Calibrator A - Store between 2 to 30°C. Contains: Buffer solution containing Na^+ , K^+ and Cl^- ions, stabilizers and preservative.

Obs.: See concentration on product label.

Number 2 - Ions Calibrator B - Store between 2 to 30°C. Contains: Buffer solution containing Na^+ , K^+ and Cl^- ions, stabilizers and preservative.

Obs.: See concentration on product label.

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TRANSPORTATION AND STORAGE CONDITIONS

The storage and transport temperature should be 2 to 30°C. Keep out of the light and avoid moisture. **Do not freeze.**

SPECIAL CARE

1- For *in vitro* diagnostic use only.

2- Strictly follow the methodology proposed to obtain exact results.

3- Water used in material cleaning must be recent and free of contaminants.

4- Do not use the product in case of change in appearance or turbidity.

5- The product may be harmful if ingested or inhaled, if swallowed seek medical help.

6- We recommend applying the local, state and federal rules for environmental protection, so that disposal of reagents and biological material can be made in accordance with current legislation.

7- To obtain information related to biosafety or in case of accidents with the product, consult the SDS (Safety Data Sheet) available on the website www.bioclin.com.br or upon request by the SAC (Advisory Service Customer) of Quibasa.

8- Do not use the product in case of damaged packaging.

9- It is essential that the instruments and equipments used are properly calibrated and subjected to periodic maintenance.

PROCESS DESCRIPTION

The use of these calibrators should be in accordance with the manufacturer's manual.

For more information, contact SAC.

Attention: After opening, the Bioions Cal ISE UR is stable for 8 weeks when stored well capped between 2 to 30°C.

For each calibration, use new aliquots of the calibrators. Do not reuse aliquots that have been exposed to air or installed in the equipment for more than 1 hour.

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QUALITY ASSURANCE

Before being released for consumption, all **Bioclin** reagents are tested by the Department of Quality Control. The quality of reagents is assured until expiration date stated on the presentation packaging, when stored and transported under appropriate conditions.

QUIBASA QUÍMICA BÁSICA Ltda

Rua Teles de Menezes, 92 - Santa Branca

CEP 31565-130 - Belo Horizonte - MG - Brasil

Tel.: +55 31 3439.5454 | E-mail: bioclin@bioclin.com.br

CNPJ: 19.400.787/0001-07 – Made in Brazil

CUSTOMER SERVICE

Customer Advisory Service

Phone.: 0800 0315454 | E-mail: sac@bioclin.com.br

ANVISA registration for Bioions Cal ISE UR kit: 10269360318

Review: August/2024

UNIVERSAL SYMBOLOGY

	CATALOG NUMBER		MADE BY
	LOT NUMBER		CONTROL
	MANUFACTURING DATE		POSITIVE CONTROL
	VALIDITY DATE (last day of the month)		NEGATIVE CONTROL
	TEMPERATURE LIMIT (store)		BIOLOGICAL RISK
	CONTENT IS SUFFICIENT FOR <N> TEST		FLAMMABLE
	SEE INSTRUCTIONS FOR USE		CORROSIVE
	IN VITRO DIAGNOSTIC PRODUCT		TOXIC
	KEEP AWAY FROM SUNLIGHT		DO NOT USE IF PACKAGE IS DAMAGED
	DO NOT REUSE		PRODUCT STERILIZED
	CAUTION		DANGER