BIOÍONS CAL ISE UR

REF B025

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FINALIDADE

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### USAGE INSTRUCTIONS

#### FUNCTION

Reagent used as calibrator in ISE modules coupled to various automatic biochemical analyzers for electrolyte ( $\text{Na}^+$ ,  $\text{K}^+$ ,  $\text{Cl}^-$ ) dosing in urine samples. For *in vitro* diagnostic use only.

#### Indicated for:

Automatic biochemistry analyzers with ISE module.

#### REAGENTS

**Number 1 - Ions Calibrator A** - Store between 2 to 30°C. Contains: Buffer solution containing  $\text{Na}^+$ ,  $\text{K}^+$  and  $\text{Cl}^-$  ions, stabilizers and preservative.

**Obs.:** See concentration on product label.

**Number 2 - Ions Calibrator B** - Store between 2 to 30°C. Contains: Buffer solution containing  $\text{Na}^+$ ,  $\text{K}^+$  and  $\text{Cl}^-$  ions, stabilizers and preservative.

**Obs.:** See concentration on product label.

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#### TRANSPORTATION AND STORAGE CONDITIONS

The storage and transport temperature should be 2 to 30°C. Keep out of the light and avoid moisture. **Do not freeze.**

#### SPECIAL CARE

**1- For *in vitro* diagnostic use only.**

**2-** Strictly follow the methodology proposed to obtain exact results.

**3-** Water used in material cleaning must be recent and free of contaminants.

**4-** Do not use the product in case of change in appearance or turbidity.

**5-** The product may be harmful if ingested or inhaled, if swallowed seek medical help.

**6-** We recommend applying the local, state and federal rules for environmental protection, so that disposal of reagents and biological material can be made in accordance with current legislation.

**7-** To obtain information related to biosafety or in case of accidents with the product, consult the MSDS (Material Safety Data Sheet) available on the website [www.bioclin.com.br](http://www.bioclin.com.br) or upon request by the SAC (Advisory Service Customer) of Quibasa.

**8-** Do not use the product in case of damaged packaging.

**9-** It is essential that the instruments and equipments used are properly calibrated and subjected to periodic maintenance.

#### PROCESS DESCRIPTION

The use of these calibrators should be in accordance with the manufacturer's manual.

For more information, contact SAC.

**Attention:** After opening, the Bioions Cal ISE UR is stable for 8 weeks when stored well capped between 2 to 30°C.

For each calibration, use new aliquots of the calibrators. Do not reuse aliquots that have been exposed to air or installed in the equipment for more than 1 hour.

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#### QUALITY ASSURANCE

Before being released for consumption, all **Bioclin** reagents are tested by the Department of Quality Control. The quality of reagents is assured until expiration date stated on the presentation packaging, when stored and transported under appropriate conditions.

**QUIBASA QUÍMICA BÁSICA Ltda**

Rua Teles de Menezes, 92 - Santa Branca

CEP 31565-130 - Belo Horizonte - MG - Brasil

Tel.: +55 31 3439.5454 | E-mail: [bioclin@bioclin.com.br](mailto:bioclin@bioclin.com.br)

CNPJ: 19.400.787/0001-07 – Made in Brazil

#### CUSTOMER SERVICE

Customer Advisory Service

Phone.: 0800 0315454 | E-mail: [sac@bioclin.com.br](mailto:sac@bioclin.com.br)

ANVISA registration for Bioions Cal ISE UR kit: 10269360318

**Review:** March/2023

UNIVERSAL SYMBOLOGY

|                                                                                     |                                          |                                                                                     |                                        |
|-------------------------------------------------------------------------------------|------------------------------------------|-------------------------------------------------------------------------------------|----------------------------------------|
|    | CATALOG NUMBER                           |    | MADE BY                                |
|    | LOT NUMBER                               |    | CONTROL                                |
|    | MANUFACTURING DATE                       |    | POSITIVE CONTROL                       |
|  | VALIDITY DATE<br>(last day of the month) |  | NEGATIVE CONTROL                       |
|  | TEMPERATURE LIMIT<br>(store)             |  | BIOLOGICAL RISK                        |
|  | CONTENT IS SUFFICIENT<br>FOR <N> TEST    |  | FLAMMABLE                              |
|  | SEE INSTRUCTIONS<br>FOR USE              |  | CORROSIVE                              |
|  | IN VITRO DIAGNOSTIC<br>PRODUCT           |  | TOXIC                                  |
|  | KEEP AWAY<br>FROM SUNLIGHT               |  | DO NOT USE IF<br>PACKAGE IS<br>DAMAGED |
|  | DO NOT REUSE                             |  | PRODUCT<br>STERILIZED                  |
|  | CAUTION                                  |  | DANGER                                 |