Bioclin

BIOLISA CMV IgG

REF K122

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- Hodinka, RL, and Friedman, HM. Human Cytomegalovirus. In: Manual of Clinical Microbiology 6th Edition (1995) 884-894.
- Hanshaw, JB, Scheiner, AP, Moxley, AW, Gaev, L, Abel, V, and Scheiner, B. School Failure and Deafness after "Silent" Congenital Cytomegalovirus Infection. N. Engl. J. Med. (1976) 295:468-470.
- Reynolds, DW, Stagno, S, Stubbs, KG, Dabte, AJ, Livingston, NM, Saxon, SS, Alford, CA. Inapparent Congenital Cytomegalovirus. N. Engl. J. Med. (1974) 790:291-296.
- Stern, H. Cytomegalovirus Vaccine: Justification and Problems. In: Waterson AP (ed.) Recent Advances in Clinical Virology (1977) 117-134.
- TELELAB. Manual de Coleta de Sangue - Diagnóstico e Monitoramento das DST, Aids e Hepatites Virais.
- QUIBASA: Dados do Departamento de Pesquisa e Desenvolvimento.

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- Hodinka, RL, and Friedman, HM. Human Cytomegalovirus. In: Manual of Clinical Microbiology 6th Edition (1995) 884-894.
- Hanshaw, JB, Scheiner, AP, Moxley, AW, Gaev, L, Abel, V, and Scheiner, B. *School Failure and Deafness after "Silent" Congenital Cytomegalovirus Infection*. N. Engl. J. Med. (1976) 295:468-470.
- Reynolds, DW, Stagno, S, Stubbs, KG, Dabte, AJ, Livingston, NM, Saxon, SS, Alford, CA. *Inapparent Congenital Cytomegalovovrus*. N. Engl. J. Med. (1974) 790:291-296.
- Stern, H. Cytomegalovirus Vaccine: Justification and Problems. In: Waterson AP (ed.) Recent Advances in Clinical Virology (1977) 117-134.
- TELELAB. Manual de Coleta de Sangue - Diagnóstico e Monitoramento das DST, Aids e Hepatites Virais.
- QUIBASA: Dados do Departamento de Pesquisa e Desenvolvimento.

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INSTRUCTIONS FOR USE

FUNCTION

Test for quantitative and qualitative of IgG antibodies for Cytomegalovirus (CMV) in human serum or plasma by enzyme immunoassay in microplate. For *in vitro* diagnostic use only.

PRINCIPLE OF ACTION

Methodology: Enzyme immunoassay or immunoenzymatic

BIOLISA CMV IgG kit is a solid phase enzyme immunoassay based on the principle of indirect qualitative and quantitative detection of IgG Antibodies to CMV in human serum, plasma and whole blood samples on filter paper. IgG Antibodies to CMV, in the sample bind to Antigen coated on the microplate forming Complexes Antigen-Antibodies IgG. After the initial incubation, the microplate is washed to remove unbound materials. Antihuman IgG Antibodies conjugated to Peroxidase are added to the microplate, which is then incubated. The Anti-human IgG Antibodies conjugated enzyme bind to IgG Antibodies present, linked to antigens. Further washing is performed to remove excess. After this step, the Substrate is added and incubated producing a blue color, which indicates the amount of IgG Antibodies to CMV in the samples. Stop Solution is added to stop the reaction having a color change from blue to yellow, measured in a microplate reader.

REAGENTS

- 1- Reference Standards (A - E)** - Store between 2 and 8°C. Contains: Five (5) flasks (A - E) of Reference Standards containing IgG Anti-CMV Antibodies in different concentrations in Buffer Solution, surfactant, stabilizers, dye and preservative. **Potentially infective.**
- The concentrations of the Reference Standards (A - E) vary with each batch. See vial label.**
- 2- Conjugate** - Store between 2 and 8°C. Contains: Buffer Solution, Human Anti-IgG Antibody linked to Peroxidase, surfactant, stabilizers, dye and preservative.
- 3- Sensitized Plate** - Store at 2 to 8°C.
- 4- Concentrated Washing** - Store at 2 and 8°C. Contains: Buffer Solution surfactant and preservative.
- 5- Sample Diluent** - Store at 2 and 8°C. Contains: Buffer Solution, stabilizers, surfactant, chelator and preservative.
- 6- Substrate** - Store at 2 and 8°C. Contains: Buffer Solution containing Urea Peroxide, Tetramethylbenzidine (TMB) and preservative.
- 7- Negative Control** - Store at 2 and 8°C. Contains: Buffer Solution, stabilizer, surfactant and preservative. **Potentially infective.**
- 8- Positive Control** - Store between 2 and 8°C. Contains: Buffer Solution, IgG Anti-CMV Antibodies, dye, stabilizers, surfactant and preservative. **Potentially infective.**
- 9- Stop Solution** - Store at 2 and 8°C. Contains: Hydrochloric Acid 1 M.
- 10- Plate Sealers.**
- 11- Filter Paper Elution Buffer** - Store at 2 and 8°C. Contains: Buffer Solution, surfactant, stabilizer and preservative.
- 12- Negative Extraction Control** - Store at 2 and 8°C. Contains: Non-reactive sample for anti-CMV IgG antibodies impregnated on Filter Paper. **Potentially infective.**
- 13- Positive Extraction Control** - Store at 2 and 8°C. Contains: Reactive sample for anti-CMV IgG antibodies impregnated in filter paper. **Potentially infective.**

PRESENTATION

REAGENTS	1	2	3
	96 Cavities	192 Cavities	480 Cavities
1 – Reference Standards (A – E)	1 Vial (A – E) x 300 µL	2 Vials (A – E) x 300 µL	5 Vials (A – E) x 300 µL
2 - Conjugate	1 Vial x 12mL	2 Vials x 12mL	5 Vials x 12mL
3 – Sensitized Plate	1 Unit (96 cavities)	2 Units (192 cavities)	5 Units (480 cavities)
4 –Concentrated Washing	1 Vial x 50mL	2 Vials x 50mL	5 Vials x 50mL
5 – Sample Diluent	1 Vial x 42mL	2 Vials x 42mL	5 Vials x 42mL
6 - Substrate	1 Vial x 12mL	2 Vials x 12mL	5 Vials x 12mL
7 –Negative Control	1 Vial x 300 µL	2 Vials x 300 µL	5 Vials x 300 µL
8 –Positive Control	1 Vial x 300 µL	2 Vials x 300 µL	5 Vials x 300 µL
9 – Stop Solution	1 Vial x 12mL	2 Vials x 12mL	5 Vials x 12mL
10 – Plate Sealers	3 Units	6 Units	15 Units
11 – Filter Paper Elution Buffer	1 Vial x 20 mL	2 Vials x 20 mL	5 Vials x 20 mL
12 – Negative Extraction Control	1 unit	2 units	5 units
13 – Positive Extraction Control	1 unit	2 units	5 units

EQUIPMENTS AND OPERATIONAL INPUTS

Materials in the kit:

- Reagents described in the above table

Required materials not contained in the kit:

- 1- Pipette capable of dispensing volumes of 5 to 300 µL with lower coefficient of variation than 1,5%.
- 2- Re-pipettor for repetitive pipetting volumes of 300 µL, with lower coefficient of variation than 1,5% or multichannel pipette (optional).
- 3- Microplate washer (optional).
- 4- ELISA reader capable of absorbance at 450 and 630 nm wavelength.
- 5- Paper towel to dry cavities
- 6- Stopwatch or watch.
- 7- Flask to store the washing solution after diluted.
- 8- Distilled water.
- 9- Tools of Quality Control.
- 10- Incubator 37°C ± 2°C.
- 11- Paper shredder (3 mm diameter) for filter paper technique.

TRANSPORTATION AND STORAGE CONDITIONS

The storage temperature should be 2 to 8°C. Transport at temperatures up to 30°C should not exceed 5 days. Keep away from light and avoid humidity. **Do not freeze.**

SPECIAL CARE

- 1- For professional *in vitro* diagnostic use only.**
- 2- Strictly follow the methodology proposed to obtain accurate results.
- 3- The sachet containing the microplate should be opened only after it reaches room temperature. Place the strip with unused cavities in the sachet, seal and store between 2 and 8°C.
- 4- The water used in material cleaning must be recent and free of contaminants.
- 5- Deionized and saturated columns release alkaline water, several ions and oxidizing and reducing agents that can significantly alter the results.
- 6- Stop Solution Contains Hydrochloric Acid, which is a strong acid. Handle it with care.

- 7- All the raw material of product is tested and should be non-reactive for HBsAg, Anti-HIV 1 & 2 and Anti HCV. However, these tests do not provide total assurance of the absence of infectious agents. The manipulation of any product containing human serum is potentially capable of transmitting diseases. Therefore, we must take due care in handling the biosafety of these products.
- 8- Always add reagents in the same order to minimize the difference in reaction time between the cavities.
- 9- As a safety measure, you should cover the plate during the reaction.
- 10- You must ensure that the bottom of the cavity is clean and dry and there are no bubbles on the surface fluid before reading the plate. Do not let the cavities run dry during the test.
- 11- Do not expose reagents, especially the Substrate, to strong light or Hypochlorite fumes during storage or incubation steps.
- 12- We recommend applying the local, state and federal rules for environmental protection, so that disposal of reagents and biological material can be made in accordance with current legislation.
- 13- To obtain information related to biosafety or in case of accidents with the product, consult the MSDS (Material Safety Data Sheet) available on the website www.bioclin.com.br or upon request by the SAC (Customer Advisory Service) of Quibasa.
- 14- Do not use the product in case of damaged packaging.
- 15- It is essential that the instruments and equipments used are properly calibrated and subjected to periodic maintenance.

SAMPLES

Serum or Plasma (EDTA or Heparin)

Hemolysed or highly lipemic samples should not be used. The samples can be kept refrigerated, between 2 and 8°C, for a maximum period of 5 days. If samples cannot be analyzed within 5 days, they can be stored for up to 30 days at -20°C.

Whole Blood on Filter Paper

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Samples dried on filter paper can be stored at room temperature as long as they are protected from direct sunlight and low humidity. For storage of up to 2 years, the samples must remain refrigerated, between 2 and 8°C.

For storage longer than 2 years, samples must be stored at a temperature of -20°C.

PROCESS DESCRIPTION

The results of the stability test show that the kit Biolisa CMV IgG is stable after opening for at least 30 days. This stability may vary according to the conditions of the test and the environment. Therefore, it is suggested to monitor the product's performance using internal kit controls and technique validation criteria.

PREPARATION OF WORKING REAGENT

Washing Solution

Dilute the contents of the Reagent N° 4 (Concentrated Washing Solution) in 1000 mL of distilled water or deionized water. After preparation the solution may be stored at 2 to 30°C until expiration date printed on the original bottle. Can be stored at room temperature. In case of crystallization, heat it at 37°C until dissolution.

Substrate

The Substrate is ready for use.

TECHNIQUE

Samples of Serum or Plasma

- Before starting the assay, place all reagents, Reference Standards (A - E), Controls and Samples to stabilize at room temperature (15 – 30°C) for at least 40 minutes.
- 1- Select the wells to be used considering: Controls and Samples (it is recommended to test in duplicate). Return the strips of the microplate will not be used Reference Standards (A – E), for the original sealed packaging.
 - 2- Select the first cavity for Blank (OPTIONAL).
 - 3- Pipette 100 µL of Sample Diluent, included in the Blank cavity.
 - 4- Pipette 5 µL of Reference Standards (A – E), Sample and Controls into previously determined wells. Observe the color change of the diluent at the time of sample addition. The color change indicates that the sample was added to the well correctly.

- 5- Homogenize gently for ± 30 seconds, cover the wells with plate sealer.
- 6- Incubate for 30 minutes ± 2 minutes in an incubator at 37°C ± 2°C.
- 7- Remove the sealing of wells.
- 8- Discard the contents of the wells by aspiration (Washer) or by decanting (manual). Use approximately 300 µl of Washing Solution, previously prepared to perform a total of five (5) washing cycles. To ensure drying of the plate, at the end of the wash, hit the board for a few seconds on absorbent paper.
- Note:** Poor washing and drying can cause inadequate results.
- 9- Pipette 100 µl of Conjugate into each well except in the Blank cavity.
- 10- Mix gently for ± 30 seconds. Cover cavities with plate sealer.
- 11- Incubate for 30 minutes ± 2 minutes in an incubator at 37°C ± 2°C.
- 12- Remove the sealer from plate cavities.
- 13- Repeat item 8.
- 14- Pipette 100 µl of Substrate into all wells, even in the cavity for White.
- 15- Shake gently for ± 30 seconds. Cover wells with plate sealer.
- 16- Incubate for 10 minutes ± 2 minutes in an incubator at 37°C ± 2°C.
- 17- Remove the sealer from plate cavities.
- 18- Pipette 100 µL of Stop Solution into all wells, even in the cavity for White.
- 19- Mix gently for ± 30 seconds.
- 20- Read using double filter: 450nm/630nm within up to 15 minutes (maximum).

Whole Blood Sample on Filter Paper

- Before starting the assay, place all reagents, Reference Standards (A - E), Controls and Samples to stabilize at room temperature (15 – 30°C) for at least 40 minutes.
- 1- Select the wells to be used considering: Reference Standards (A – E), Controls and Samples (it is recommended to test in duplicate). Return the strips of the microplate will not be used for the original sealed packaging.
 - 2- Select the first cavity for Blank (OPTIONAL).
 - 3- Add a 3 mm disk of Positive Extraction Control, Negative Extraction Control and Whole Blood Sample on Filter Paper, previously perforated, in the determined wells.
 - 4- Pipette 100 µL of Elution Buffer in Filter Paper, included in the Blank cavity.
 - 5- Pipette 5 µL of Reference Standards (A – E) and Controls into previously determined wells.
 - 6- Homogenize gently for ± 30 seconds, cover the wells with plate sealer.
 - 7- Incubate for 30 minutes ± 2 minutes in an incubator at 37°C ± 2°C.
 - 8- Remove the sealing of wells.
 - 9- Discard the contents of the wells by decanting (manual). Remove the paper discs, if necessary, with the aid of a needle or tip.
 - Use approximately 300 µl of Washing Solution, **previously prepared** to perform a total of five (5) washing cycles. To ensure drying of the plate, at the end of the wash, hit the board for a few seconds on absorbent paper.
 - Note:** Poor washing and drying can cause inadequate results.
 - 10- Pipette 100 µl of Conjugate into each well, included in the Blank cavity.
 - 11- Mix gently for ± 30 seconds. Cover cavities with plate sealer.
 - 12- Incubate for 30 minutes ± 2 minutes in an incubator at 37°C ± 2°C.
 - 13- Remove the sealer from plate cavities.
 - 14- Repeat item 9.
 - 15- Pipette 100 µl of Substrate into all wells, even in the cavity for White.
 - 16- Shake gently for ± 30 seconds. Cover wells with plate sealer.
 - 17- Incubate for 10 minutes ± 2 minutes in an incubator at 37°C ± 2°C.
 - 18- Remove the sealer from plate cavities.
 - 19- Pipette 100 µL of Stop Solution into all wells, even in the cavity for White.
 - 20- Mix gently for ± 30 seconds.
 - 21- Read using double filter: 450nm/630nm within up to 15 minutes (maximum).