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USAGE INSTRUCTIONS

FUNCTION

Anticoagulant solution for hematologic tests in biological samples of blood. For *in vitro* diagnostic use only.

PRINCIPLE OF ACTION

EDTA (Ethylenediamine Tetraacetic Acid) is used to obtain total blood or plasma. Operates blocking ionized Calcium by chelation leading to formation of an insoluble Ca-EDTA complex.

REAGENTS

EDTA Anticoagulant - Store between 15 and 30°C. Contains: EDTA < 700 mmol/L and dye.

PRESENTATION

Presentation	Volume
1	20 mL
2	250 mL

EQUIPMENTS AND OPERATIONAL INPUTS

Test tubes and other materials needed to collect blood. They can be found at markets specialized on Laboratories of Clinical Analysis.

TRANSPORTATION AND STORAGE CONDITIONS

The storage and transportation temperatures should be between 15 to 30°C. Protect from light and avoid moisture.

SPECIAL CARE

- 1- For *in vitro* diagnostic use only.
- 2- Water used in material cleaning must be recent and free of contaminants.
- 3- Saturated deionizer columns release alkaline water, many ions, oxidizing agents and reducers that may alter the results significantly.
- 4- EDTA should not be used in dosages of Calcium and determinations of prothrombin time.
- 5- We recommend applying the local, state and federal rules for environmental protection, so that disposal of reagents and biological material can be made in accordance with current legislation.
- 6- To obtain information related to biosafety or in case of accidents with the product, consult the MSDS (Material Safety Data Sheet) available on the website www.bioclin.com.br or upon request by the SAC (Customer Advisory Service) of Quibasa.

7- Do not use the product in case of damaged packaging.

8- It is essential that the instruments and equipments used are properly calibrated and subjected to periodic maintenance.

SAMPLES

Peripheral blood. Homogenize blood smoothly to avoid hemolyses.

TECHNIQUE

Use 1 drop of anticoagulant for each 5 mL of blood. Smoothly homogenize through inversion. Separate plasma by centrifuging (3,000 rpm), for 5 minutes. Transfer supernatant to a test tube and maintain plasma under refrigeration up to the moment of usage.

PROCEDURE LIMITATIONS

The volume ratio of anticoagulant/blood must be precise, for morphological preservation of cellular elements.

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QUALITY ASSURANCE

Before being released for consumption, all **Bioclin** reagents are tested by the Department of Quality Control. The quality of reagents is assured until expiration date stated on the presentation packaging, when stored and transported under appropriate conditions.

QUIBASA QUÍMICA BÁSICA Ltda

Rua Teles de Menezes, 92 - Santa Branca
 CEP 31565-130 - Belo Horizonte - MG - Brasil
 Phone.: +55 31 3439.5454 | E-mail: bioclin@bioclin.com.br
 CNPJ: 19.400.787/0001-07 - Made in Brazil

CUSTOMER SERVICE

Customer Advisory Service
 Phone.: 0800 0315454 | E-mail: sac@bioclin.com.br

ANVISA registration for EDTA kit: 10269360088

Review: July/2022

UNIVERSAL SYMBOLOGY

	CATALOG NUMBER		WIPCE BY
	LOT NUMBER		CONTROL
	MANUFACTURING DATE		POSITIVE CONTROL
	VALIDITY DATE (last day of the month)		NEGATIVE CONTROL
	TEMPERATURE LIMIT (store)		BIOLOGICAL RISK
	CONTENT IS SUFFICIENT FOR -NO- TEST		FLAMMABLE
	SEE INSTRUCTIONS FOR USE		CORROSIVE
	IN VITRO DIAGNOSTIC PRODUCT		TOXIC
	KEEP AWAY FROM SUNLIGHT		DO NOT USE IF PACKAGE IS DAMAGED
	DO NOT REUSE		PRODUCT STERILIZED
	CAUTION		DANGER