BIOLISA TOXOPLASMOSE IgG

REF K127

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FINALIDADE

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1. Feldman, HA. Toxoplasmosis: An Overview. Acad. Med. (1974) 50:110-127.
2. Remington, JS. Toxoplasmosis in the Adult. Acad. Med. (1974) 50:211-227.
3. McLeod, R, and Remington JS. In Harrison.s Principles of Internal Medicine. (1980) 879-885.
4. Frenkel, JK. Toxoplasma in and Around Us. Bioscience. (1973) 23:343-352.
5. Feldman, HA. Epidemiology of Toxoplasma Infections, Epid. Rev. (1982) 4:204-213.
6. Krick, JA, and Remington, JS. Toxoplasmosis in the Adult – An Overview. N. Engl. J. Med. (1978) 298(10):550-553.
7. Bryan, RT, and Wilson, M. Toxoplasmosis. Lab Management (1988) 26:40-43..
8. Bioclin – Dados de arquivos

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REF K127

USAGE INSTRUCTIONS

FUNCTION
Test for quantitative and qualitative of IgG antibodies to *Toxoplasma gondii* in serum or human plasma by enzyme immunoassay in microplate. For *in vitro* diagnostic use only.

PRINCIPLE OF ACTION
Methodology: Enzyme immunoassay or immunoenzymatic
BIOLISA Toxoplasmosis IgG kit is a solid phase enzyme immunoassay based on the principle of indirect qualitative and quantitative detection of IgG Antibodies to *Toxoplasma gondii* in human serum or plasma. IgG Antibodies to *Toxoplasma gondii*, in the sample bind to Antigen coated on the microplate forming Complexes Antigen-Antibodies IgG. After the initial incubation, the microplate is washed to remove unbound materials. Anti-human IgG Antibodies conjugated to Peroxidase are added to the microplate, which is then incubated. The Anti-human IgG Antibodies conjugated enzyme bind to IgG Antibodies present, linked to antigens. Further washing is performed to remove excess. After this step, the Substrate is added and incubated producing a blue color, which indicates the amount of IgG Antibodies to *Toxoplasma gondii* in the samples. Stop Solution is added to stop the reaction having a color change from blue to yellow, measured in a microplate reader.

REAGENTS
1- Reference Standards (A - D) - Store between 2 and 8°C. Contains: Four (4) bottle (A - D) Reference Standards containing IgG Antibodies Anti-*Toxoplasma gondii* at different concentrations in Buffer Solution and preservative. **Potentially infectious.**
Reference Standard (A - D) concentration varies with the lot. See bottle label.
2- Conjugate - Store between 2 and 8°C. Contains: Human Anti-IgG Antibody linked to Peroxidase and preservative.
3- Sensitized Plate - Store between 2 and 8°C
4- Concentrated Washing - Store between 2 and 8°C. Contains: Buffer Solution, surfactant and preservative.
5- Sample Diluent - Store between 2 and 8°C. Contains: Buffer Solution and preservative.
6- Substrate - Store between 2 and 8°C. Contains: Buffer Solution containing Urea Peroxide, Tetramethylbenzidine (TMB) and preservative.
7- Negative Control - Store between 2 and 8°C. Contains: Non-reactive IgG Antibodies to *Toxoplasma gondii* and preservative. **Potentially infectious.**
8- Positive Control - Store between 2 and 8°C. Contains: IgG Antibodies Anti-*Toxoplasma gondii* and preservative. **Potentially infectious.**
9- Stop Solution - Store between 2 and 8°C. Contains: Chloridric Acid 1 M.
10- Plate Sealers

REAGENTS	1	2	3
	96 CAVITIES	192 CAVITIES	480 CAVITIES
1- Reference Standards (A - D)	1 Flask (A - D) x 100 µL	2 Flasks (A - D) x 100 µL	5 Flasks (A - D) x 100 µL
2- Conjugate	1 Flask x 12 mL	2 Flasks x 12 mL	5 Flasks x 12 mL
3- Sensitized Plate	1 unit (96 cavities)	2 units (96 cavities)	5 units (96 cavities)
4- Concentrated Washing	1 Flask x 50 mL	2 Flasks x 50 mL	5 Flasks x 50 mL
5- Sample Diluent	1 Flask x 42 mL	2 Flasks x 42 mL	5 Flasks x 42 mL
6- Substrate	1 Flask x 11 mL	2 Flasks x 11 mL	5 Flasks x 11 mL
7- Negative Control	1 Flask x 100 µL	2 Flasks x 100 µL	5 Flasks x 100 µL
8- Positive Control	1 Flask x 100 µL	2 Flasks x 100 µL	5 Flasks x 100 µL
9- Stop Solution	1 Flask x 12 mL	2 Flasks x 12 mL	5 Flasks x 12 mL
10- Plate Sealers	3 Units	6 Units	15 Units

EQUIPMENTS AND OPERATIONAL INPUTS
Materials in the kit:
- Reagents described in the above table
- Operating instructions (manual)

Required materials not contained in the kit:
1- Pipette capable of dispensing volumes of 5 and 100 µL with lower coefficient of variation than 1,5%.
2- Re-pipettor for repetitive pipetting volumes of 100 µL, with lower coefficient of variation than 1,5% or multichannel pipette (optional).
3- Microplate washer (optional).
4- ELISA reader capable of absorbance at 450 and 630 nm wavelength.
5- Paper towel to dry cavities
6- Stopwatch or watch.
7- Flask to store the washing solution after diluted.
8- Distilled water.
9- Tools of Quality Control.
10- Incubator 37°C ± 2°C.

TRANSPORTATION AND STORAGE CONDITIONS
The storage temperature should be 2 to 8°C. The transport can be done under ambient temperature (up to 30°C) for up to 72 (seventy two) hours. Keep away from light and avoid moisture. **Do not freeze.**

SPECIAL CARE
1- For professional *in vitro* diagnostic use only.
2- Strictly follow the methodology proposed to obtain accurate results.
3- The sachet containing the microplate should be opened only after it reaches room temperature. Place the strip with unused cavities in the sachet, seal and store at 2 - 8°C.
4- The water used in material cleaning must to be recent and free of contaminants.
5- Deionized and saturated columns release alkaline water, several ions and oxidizing and reducing agents that can significantly alter the results.
6- Stop Solution contains Hydrochloric Acid which is a strong acid. Handle it with care.
7- All the raw material of product is tested and should be nonreactive for HBsAg, Anti-HIV 1&2 and Anti-HCV. However, these tests do not provide total assurance of the absence of infectious agents. The manipulation of any product containing human serum is potentially capable of transmitting diseases. Therefore, we must take due care in handling the bio safety of these products.

8- Always add reagents in the same order to minimize the difference in reaction time between the cavities.
9- As a safety measure, you should cover the plate during the reaction.
10- You must ensure that the bottom of the cavity is clean and dry, and there are no bubbles on the surface fluid before reading the plate. Do not let the cavities run dry during the test;
11- Do not expose reagents, especially the Substrate, to strong light or Hypochlorite fumes during storage or incubation steps;
12- We recommend applying the local, state and federal rules for environmental protection, so that disposal of reagents and biological material can be made in accordance with current legislation.
13- To obtain information related to biosafety or in case of accidents with the product, consult the MSDS (Material Safety Data Sheet) available on the website www.bioclin.com.br or upon request by the SAC (Customer Advisory Service) of Quibasa.
14- Do not use the product in case of damaged packaging.
15- It is essential that the instruments and equipments used are properly calibrated and subjected to periodic maintenance.

SAMPLES
Use serum or plasma (EDTA or Heparin).
Hemolyzed or highly lipemic samples should not be used.
Samples may be refrigerated at between 2 and 8°C for a maximum of 5 days. If samples can not be analyzed within 5 days, they can be stored for up to 30 days at -20°C (freezer).

PROCESS DESCRIPTION

PREPARATION OF WORKING REAGENTS
Washing Solution
Dilute the contents of the Flask N°4 (Concentrated Washing) in 1000 mL of distilled or deionized water. After preparation the solution may be stored at 2 to 30°C until expiration date printed on the original bottle. Can be stored at room temperature. In case of crystallization, heat it at 37°C until dissolution.

Substrate
The Substrate is ready for use.

TECHNIQUE
Before starting the assay, place all reagents, Reference Standards (A - D), Controls and Samples to stabilize at room temperature (15 – 30°C) for at least 40 minutes.
1- Separate the cavities to be used considering: Reference Standards (A - D), Controls and Samples (it is recommended to test in duplicate). Return the unused strips from the microplate to the original sealed packaging.
2- Separate the first cavity to the White (OPTIONAL).
3- Pipette 100 µL of Sample Diluent into the previously determined cavities.
4- Pipette 5 µL of Reference Standards (A - D), Controls and Samples into the previously determined cavities. Observe the color change of the diluent at the time of sample addition. The color change indicates that the sample was added to the cavities correctly.
5- Homogenize gently for ± 30 seconds, cover the cavities with plate sealer.
6- Incubate for 30 minutes ± 2 minutes in an incubator at 37°C ± 2°C.
7- Remove the plate sealer from the cavities.
8- Discard the contents of the cavities by aspiration (Washer) or by decanting (Manual). Use approximately 300 µL of Washing Solution, **previously prepared**, to perform a total of five (5) washing cycles. To ensure drying of the plate at the end of the wash, hit the board for a few seconds on absorbent paper.
Note: Poor washing/drying can cause inadequate results.
9- Pipette 100 µL of Conjugate into each cavities except in the Blank cavity (OPTIONAL).
10- Homogenize gently for ± 30 seconds. Cover the cavities with the plate sealer.
11- Incubate for 30 minutes ± 2 minutes in an incubator at 37°C ± 2°C.
12- Remove the plate sealer from the cavities.
13- Repeat item 8.
14- Pipette 100 µL of Substrate into all cavities.

15- Homogenize gently for ± 30 seconds. Cover the cavities with the plate sealer.
16- Incubate for 10 minutes ± 2 minutes in an incubator at 37°C ± 2°C.
17- Remove the plate sealer from the cavities.
18- Pipette 100 µL Stop Solution into all cavities.
19- Homogenize gently for ± 30 seconds.
20- Read using double filter: 450 nm / 630 nm in up to 15 minutes (maximum).

TECHNIQUE VERIFICATION
Verify if the results obtained by the reading of the Blank, Reference Standards (A - D) and Controls are compatible to the values below:

ITEM	ABSORBANCE
Blank	< 0,100
Reference Standard A	< 0,100
Reference Standard B	> 0,2 e < 0,6
Reference Standard C	> Abs B e < Abs D
Reference Standard D	> 1,4
Positive Control	> 1,0
Negative Control	< 0,100

If the values are out of the expected values, you must repeat the technique.

CALCULATIONS

QUALITATIVE
Cut Off regarded as the mean absorbance obtained with the Reference Standard B.
Example:

ITEM	ABSORBANCE
Reference Standard B	A1 = 0,356
	A2 = 0,352
Cut Off = Average Absorbance from Reference Standard B	Cut Off = (0,356 + 0,352) / 2 Cut Off = 0,354

Calculate the index by dividing the Sample absorbance by the value of the Cut Off.
Example:

ITEM	ABSORBANCE
Sample	0,933
Cut Off Value	0,354
Index = Sample / Cut Off Value	Index = 0,933 / 0,354 Index = 2,635

Note: The data presented in the examples are for illustration only and can not be used to calculate the results.
QUANTITATIVE
A calibration curve is used to determine the concentration of anti-*Toxoplasma gondii* antibodies in unknown samples.

Preparation of Calibration Curve
Record the absorbance obtained in the microplate Reader as outlined in example 1. Calculate the averages of duplicates (if they are carried duplicates). Plot the absorbances of each Reference Standard *versus* the corresponding concentration in UI/mL on graph paper (before you plot them on the graph.) Plot the curve.

Example 1

STANDARDS	ABSORBANCE	AVERAGE ABSORBANCE	CONCENTRATION
A	0,006	0,006	0,0
	0,007		
B	0,377	0,372	32,0
	0,367		
C	0,872	0,872	75,0
	0,873		
D	1,851	1,837	300,0
	1,824		

REPEATABILITY		SAMPLE		
		1	2	3
Average		0,894	1,658	0,076
Standard Deviation		0,096	0,027	0,008
Coefficient of Variation (%)		5,105	1,642	11,058

REPRODUCIBILITY

The reproducibility was calculated from 10 successive determinations for 3 consecutive days, using 3 samples with different values, obtaining the following absorbance results:

REPRODUCIBILITY		SAMPLE		
		1	2	3
Average		0,844	1,639	0,078
Standard Deviation		0,092	0,022	0,012
Coefficient of Variation (%)		5,485	1,468	12,74

Samples that have absorbance above the Reference Standard D, must be pre-diluted using Sample Diluent and should be retested. The concentration must be multiplied by the dilution factor. Automatic reading and calculation can be performed through the point to point function in appropriate programs of computer.

Note: The data presented in the examples are for illustration only and may not be used in lieu of the calibration curve, which must be constructed in the laboratory.

RESULTS

RESULTS	QUALITATIVE	QUANTITATIVE
	INDEX	CONCENTRATION
Negative	≤ 0,90	≤ 28
Positive	≥ 1,00	≥ 32
Undetermined	0,91 - 0,99	29 - 31

Note: In case of indeterminate results, the sample must be retested. Samples that results have been obtained repeatedly indeterminate should be retested using an alternative method. If the results remain uncertain, one must collect a new sample in two weeks. Should prevail the result of the last sample collected.

The results provided by this kit must be interpreted by the medical professional responsible, not being the only criterion for the determination of diagnosis and/or treatment of the patient.

PROCEDURE LIMITATIONS

The interpretation of a diagnostic test there should not be based on a single run. This should include confirmation of other tests before a sample is considered positive. A negative result does not exclude the possibility of exposure. All results should be interpreted in conjunction with other clinical information available before the descriptive diagnosis of the disease.

INTERNAL QUALITY CONTROL

The Clinical Laboratory must have an internal quality control, where all procedures, rules, limits and tolerance to variations be clearly established. It is important to mention that all measurement systems present a analytical variety, and it must be monitor by the laboratory. Therefore, it is recommendable the use of controls, allowing the precision and accuracy of the dosages.

PRODUCT PERFORMANCE

QUALITY CONTROL

Accuracy

REPEATABILITY

The repeatability was calculated from 10 successive determinations, using 3 samples with different values, obtaining the following absorbance results:

REPEATABILITY	SAMPLE		
	1	2	3
Average	0,894	1,658	0,076
Standard Deviation	0,096	0,027	0,008
Coefficient of Variation (%)	5,105	1,642	11,058

REPRODUCIBILITY

The reproducibility was calculated from 10 successive determinations for 3 consecutive days, using 3 samples with different values, obtaining the following absorbance results:

REPRODUCIBILITY		SAMPLE		
		1	2	3
Average		0,844	1,639	0,078
Standard Deviation		0,092	0,022	0,012
Coefficient of Variation (%)		5,485	1,468	12,74

Clinical sensitivity and specificity

BIOLISA Toxoplasmosis IgG Kit analyzed clinical samples in comparison with other methods of EIA. The results show that the clinical sensitivity of the BIOLISA Toxoplasmosis IgG kit is 99,24% and clinical specificity is > 99,99%.

	Expected Results	Biolisa Toxoplasmosis IgG
Positive Sample	131	130
Negative Sample	68	68
Total Tested Sample	199	

Clinical Sensitivity: 99,24% (130/131)
Clinical Specificity: > 99,99% (68/68)

Linearity

The reaction is able to detect concentrations up to the highest point on the calibration curve. For samples with higher values, dilute the same with Sample Diluent, repeat the dosage and multiply the results by the dilution factor.

DIAGNOSTIC SIGNIFICANCE

Toxoplasma gondii is the causative agent of toxoplasmosis. It is an obligate intracellular protozoan that has been found in many species of birds, reptiles and mammals. The agent can be transmitted through organ transplants, blood transfusions and leukocyte, contact with cat feces contaminated and ingestion of contaminated raw meats. In adults, the infection is usually benign or asymptomatic. However, symptomatic cases, including fatal cases occur in patients immunosuppressed patients who have clinical or laboratory evidence of central nervous system damage. In children, the risk of fetal infection varies according to the time of pregnancy when the mother is infected. In maternal infections that occur during the first quarter, the probability of infection through to the fetus is smaller. However, if transmission occurs, severe outcomes such as miscarriage and hydrocephalus are more likely. Infections acquired later in pregnancy, where the transmissions fetal occur most frequently tend to be less severe, but even they can generate congenital manifestations, including brain calcification and impaired learning. After infection, IgM antibodies appear 5 days and had reduced levels within a few weeks or months. IgG antibodies usually appear 1 - 2 weeks after infection, reaching levels peak at 6 - 10 weeks persisting for life.

NUMBER OF TESTS

Presentation 1 - 96 tests
Presentation 2 - 192 tests
Presentation 3 - 480 tests

BIBLIOGRAPHIC REFERENCES

1. Feldman, HA. Toxoplasmosis: An Overview. Acad. Med. (1974) 50:110-127.
2. Remington, JS. Toxoplasmosis in the Adult. Acad. Med. (1974) 50:211-227.
3. McLeod, R, and Remington JS. In Harrison.s Principles of Internal Medicine. (1980) 879-885.
4. Frenkel, JK. *Toxoplasma in and Around Us*. Bioscience. (1973) 23:343-352.
5. Feldman, HA. Epidemiology of Toxoplasma Infections, Epid. Rev. (1982) 4:204-213.
6. Krick, JA, and Remington, JS. *Toxoplasmosis in the Adult – An Overview*. N. Engl. J. Med. (1978) 298(10):550-553.
7. Bryan, RT, and Wilson, M. *Toxoplasmosis*. Lab. Management (1988) 26:40-43.
8. Bioclin – Dados de arquivos.

QUALITY ASSURANCE

Before being released for consumption, all **Bioclin** reagents are tested by the Department of Quality Control. The quality of reagents is assured until expiration date stated on the presentation packaging, when stored and transported under appropriate conditions.

QUIBASA QUÍMICA BÁSICA Ltda
Rua Teles de Menezes, 92 - Santa Branca
CEP 31565-130 - Belo Horizonte - MG - Brasil
Phone: +55 (31) 3439.5454 - Fax: +55 (31) 3439.5455
E-mail: bioclin@bioclin.com.br
CNPJ: 19.400.787/0001-07 - Made in Brazil

CUSTOMER SERVICE

Customer Advisory Service
Phone.: 0800 0315454
E-mail: sac@bioclin.com.br

ANVISA registration for Biolisa Toxoplasmosis IgG kit: 10269360196

Review: February/2019

English 2/2

UNIVERSAL SYMBOLOGY

CATALOG NUMBER

BATCH CODE

DATE OF MANUFACTURE

USED BY
(last day of month)

TEMPERATURE LIMITATION
(store at)

CONTAINS SUFFICIENT
FOR <N> TESTS

CONSULT INSTRUCTIONS
FOR USE

IN VITRO DIAGNOSTIC DEVICE

EUROPEAN AUTHORIZED
REPRESENTATIVE

KEEP AWAY
FROM SUNLIGHT

MANUFACTURED BY

CONTROL

POSITIVE CONTROL

NEGATIVE CONTROL

BIOLOGICAL RISK

INFLAMMABLE

CORROSIVE

POISON

CE MARK

DO NOT USE IF
PACKAGE IS
DAMAGED