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EDTA**REF A004****USAGE INSTRUCTIONS****FUNCTION**

Anticoagulant solution for hematologic tests. For *in vitro* diagnostic use only.

PRINCIPLE OF ACTION

EDTA (Ethylenediamine Tetraacetic Acid) is used to obtain total blood or plasma. Operates blocking ionized Calcium by chelation leading to formation of an insoluble Ca-EDTA complex.

REAGENTS

EDTA Anticoagulant - Store between 15 and 30°C.
Contains: Sodium EDTA 300 mmol/L.

PRESENTATION

Presentation	Volume
1	20 mL
2	250 mL

EQUIPMENTS AND OPERATIONAL INPUTS

Test tubes and other materials needed to collect blood. They can be found at markets specialized on Laboratories of Clinical Analysis.

TRANSPORTATION AND STORAGE CONDITIONS

The storage and transportation temperatures should be between 15 to 30°C. Protect from light and avoid moisture.

SPECIAL CARE

- 1- For professional *in vitro* diagnostic use only.**
- 2- Water used in material cleaning must be recent and free of contaminants.**
- 3- Saturated deionizer columns release alkaline water, many ions, oxidizing agents and reducers that may alter the results significantly.**
- 4- EDTA should not be used in dosages of Calcium and determinations of prothrombin time.**
- 5- We recommend applying the local, state and federal rules for environmental protection, so that disposal of reagents and biological material can be made in accordance with current legislation.**
- 6- To obtain information related to biosafety or in case of accidents with the product, consult the MSDS (Material Safety Data Sheet) available on the website www.bioclin.com.br or upon request by the SAC (Customer Advisory Service) of Quibasa.**
- 7- Do not use the product in case of damaged packaging.**
- 8- It is essential that the instruments and equipments used are properly calibrated and subjected to periodic maintenance.**

SAMPLES

Peripheral blood. Homogenize blood smoothly to avoid hemolyses.

TECHNIQUE

Use 1 drop for each 5 mL of blood.

PROCEDURE LIMITATIONS

The volume ratio of anticoagulant/blood must be precise, for morphological preservation of cellular elements.

NUMBER OF TESTS

A 004-1..... 400 tests / 50 µL of Reagent
A 004-2..... 5000 tests / 50 µL of Reagent

QUALITY ASSURANCE

Before being released for consumption, all **Bioclin** reagents are tested by the Department of Quality Control. The quality of reagents is assured until expiration date stated on the presentation packaging, when stored and transported under appropriate conditions.

**QUIBASA QUÍMICA BÁSICA Ltda**

Rua Teles de Menezes, 92 - Santa Branca
CEP 31565-130 - Belo Horizonte - MG - Brasil
Phone.: +55 (31) 3439.5454 - Fax: +55 (31) 3439.5455
E-mail: bioclin@bioclin.com.br
CNPJ: 19.400.787/0001-07 - Made in Brazil

EC	REP	OBELIS S.A.
----	-----	--------------------

Bd. Général Wahis, 53
1030 Brussels, Belgium

CUSTOMER SERVICE

Customer Advisory Service
Phone.: 0800 0315454
E-mail: sac@bioclin.com.br

ANVISA registration for EDTA kit: 10269360088

Review: June/2013

UNIVERSAL SYMBOLOGY

	CATALOG NUMBER		MANUFACTURED BY
	BATCH CODE		CONTROL
	DATE OF MANUFACTURE		POSITIVE CONTROL
	USED BY (last day of month)		NEGATIVE CONTROL
	TEMPERATURE LIMITATION (store at)		BIOLOGICAL RISK
	CONTAINS SUFFICIENT FOR <N> TESTS		INFLAMMABLE
	CONSULT INSTRUCTIONS FOR USE		CORROSIVE
	IN VITRO DIAGNOSTIC DEVICE		POISON
	EUROPEAN AUTHORIZED REPRESENTATIVE		CE MARK
	KEEP AWAY FROM SUNLIGHT		DO NOT USE IF PACKAGE IS DAMAGED