## Fe AUTOMAÇÃO

REF K209

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## FINALIDADE

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### USAGE INSTRUCTIONS

#### FUNCTION

Method for determination of Iron Serum. Colorimetric test, for *in vitro* diagnostic use only.

#### PRINCIPLE OF ACTION

**Methodology:** Modified Goodwin/Ferrozine  
Iron is released from Transferrin in an acid environment and lowered to its ferrous state by the action of Hydroxylamine. Subsequently, reacts with Ferrozine leading to the formation of a violet colored complex.

#### REAGENTS

**Number 1 - Reduction Buffer** - Store between 2 and 8°C. Contains: Sodium Acetate < 900 mmol/L, Hydroxylamine < 800 mmol/L, surfactants and stabilizer.

**Number 2 - Color Reagent** - Store between 2 and 8°C. Contains: Ferrozine < 25 mmol/L, Hydroxylamine < 20 mmol/L and stabilizers.

**Number 3 - Standard** - Store between 2 and 8°C. Contains: Iron 200 µg/dL, stabilizer and preservative.

#### PRESENTATION

Presentation	Reagent N° 1	Reagent N° 2	Reagent N° 3
1	1 x 40 mL	1 x 10 mL	1 x 3 mL
2	2 x 40 mL	2 x 10 mL	1 x 3 mL
3	4 x 40 mL	4 x 10 mL	1 x 3 mL
4	6 x 40 mL	6 x 10 mL	1 x 3 mL
5	8 x 40 mL	8 x 10 mL	1 x 3 mL
6	2 x 40 mL	1 x 20 mL	1 x 3 mL
7	4 x 40 mL	2 x 20 mL	1 x 3 mL
8	1 x 20 mL	1 x 5 mL	1 x 3 mL
9	2 x 20 mL	2 x 5 mL	1 x 3 mL
10	3 x 20 mL	3 x 5 mL	1 x 3 mL
11	4 x 20 mL	4 x 5 mL	1 x 3 mL
12	5 x 20 mL	5 x 5 mL	1 x 3 mL

#### EQUIPMENTS AND OPERATIONAL INPUTS

Biochemistry automated equipment or semi-automated, water bath, pipettes, test tubes, Biocontrol N and Biocontrol P Bioclin. They can be found at markets specialized on Laboratories of Clinical Analysis.

#### TRANSPORTATION AND STORAGE CONDITIONS

Store temperatures must be between 2 and 8°C. Transportation in temperatures between 15 and 30°C must not exceed 72 (seventy two) hours. Keep away from light and avoid humidity. **Do not freeze.**

#### SPECIAL CARE

- 1- For professional *in vitro* diagnostic use only.
- 2- Strictly follow the methodology proposed to obtain exact results.
- 3- Water used in material cleaning must be recent and free of contaminants.
- 4- Saturated deionizer columns release alkaline water, many ions, oxidizing agents and reducers that may alter the results significantly.

5- Hemolysis, even if discrete, can interfere with dosage.

6- Do not use plasma.

7- Every material used for analyses must be previously emerged in a recent HCL (7%) solution for 30 minutes. Rinse it with deionized water.

8- We recommend applying the local, state and federal rules for environmental protection, so that disposal of reagents and biological material can be made in accordance with current legislation.

9- To obtain information related to biosafety or in case of accidents with the product, consult the MSDS (Material Safety Data Sheet) available on the website [www.bioclin.com.br](http://www.bioclin.com.br) or upon request by the SAC (Customer Advisory Service) of Quibasa.

10- Do not use the product in case of damaged packaging.

11- It is essential that the instruments and equipments used are properly calibrated and subjected to periodic maintenance.

#### SAMPLES

Serum obtained free of hemolysis, stable for 21 days between 2 and 8°C and 1 year at -20°C.<sup>7</sup>

#### PROCESS DESCRIPTION

#### TECHNIQUE

Bioclin recommends, as control serum, Biocontrol N and P Bioclin Kits.

Mark 3 test tubes with letters B (Blank), P (Standard), A (Sample) and proceed as follows:

	Blank	Standard	Sample
Reagent N°1	0,8 mL	0,8 mL	0,8 mL
Distilled Water	0,1 mL	----	----
Sample	----	----	0,1 mL
Reagent N°3	----	0,1 mL	----

Smoothly homogenize and determine the absorbances of the Sample and Standard at 560 nm (540 - 580 nm), hitting the zero with the distilled water. This will be the absorbance  $A_1$ .

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Smoothly homogenize and incubate in water-bath at 37°C, per 5 minutes. Determinate the absorbances of Sample and the Standard in 560 nm (540 - 580 nm), matching the zero with the Blank. This will be the absorbance of the sample  $A_2$ .

#### CALCULATIONS

Due to the dilution, the  $A_1$  reading of the Standard and Sample should be corrected for the final volume, using the factor of 0.82. Thus,  $A_{1CORRECTED}$  is obtained.

$$\text{Standard-}A_{1CORRECTED} = \text{Standard } A_1 \times 0.82$$

$$\text{Sample-}A_{1CORRECTED} = \text{Sample } A_1 \times 0.82$$

$$\text{Iron Serum } (\mu\text{g/dL}) = \frac{\text{Sample } (A_2 - A_{1CORRECTED}) \times 200}{\text{Standard } (A_2 - A_{1CORRECTED})}$$

The color reaction follows the Lambert Beer Law, it can use the Calibration Factor:

$$\text{Calibration Factor} = \frac{200}{\text{Standard } (A_2 - A_{1CORRECTED})}$$

$$\text{Iron Serum } (\mu\text{g/dL}) = \text{Sample } (A_2 - A_{1CORRECTED}) \times \text{Calibration Factor}$$

The results are expressed in µg/dL.

#### PROCEDURE LIMITATIONS

The use of water contaminated by Iron increase white color intensity, changing the results.

#### INTERFERENCE

Triglycerides up to 1000 mg/dL, Bilirubin up to 20 mg/dL and Copper up to 7,5 mg/dL did not interfere with the results.

Some drugs can raise levels of Serum Iron such as Chloramphenicol, Estrogens, Ethanol, Methotrexate and Anovulatory. Other drugs may decrease the levels of Serum Iron such as: Acetylsalicylic Acid (high doses), Allopurinol, Cortisone and Corticotrophin.

Samples of patients on hemodialysis, using Heparin, or with presence of fibrinogen are not interfered with in the analysis.

#### INTERNAL QUALITY CONTROL

The Clinical Laboratory must have an internal quality control, where all procedures, rules, limits and tolerance to variations be clearly established. It is important to mention that all measurement systems present a analytical variety, and it must be monitor by the laboratory. Therefore, it is recommendable the use of controls, allowing the precision and accuracy of the dosages.

#### TRACEABILITY

The kit's standard is traceable to the reference material NIST (National Institute of Standards and Technology) SRM 937.

#### REFERENCE VALUES

The reference values in µg/dL, for this method, were obtained through the determination of Iron in healthy populations of male and female.

Iron Serum (µg/dL)		
Newborn		100 - 250
Infant		40 - 100
Children		50 - 120
Adult	Male	65 - 170
	Female	50 - 170

To convert the values from µg/dL to mmol/L, multiply by 0,179.

These values should be used as guidance, and each laboratory should establish its range of reference values, according to the population served.

The results provided by this kit must be interpreted by the medical professional responsible, not being the only criterion for the determination of diagnosis and/or treatment of the patient.

## PRODUCT PERFORMANCE

### QUALITY CONTROL

#### Accuracy

#### COMPARISON OF METHODS AND METHODOLOGICAL SPECIFICITY

The Fe Automated kit was compared with other method for determination of Iron dosage commercially available. 42 analyzes were performed and the results were evaluated. The linear equation obtained was  $Y = 0,9744X + 1,6825$  and correlation coefficient 0,999. With these results we can conclude that the kit shows good methodological specificity.

#### Accuracy

#### REPEATABILITY

The repeatability was calculated from 40 successive determinations, using 3 samples with different concentrations, obtaining the following results:

	Sample 1	Sample 2	Sample 3
<b>Concentration Promedio (<math>\mu\text{g/dL}</math>)</b>	72,90	222,73	27,90
<b>Standard Deviation (<math>\mu\text{g/dL}</math>)</b>	1,43	0,93	0,63
<b>Coefficiente de Variación (%)</b>	1,96	0,42	2,27

#### REPRODUCIBILITY

The reproducibility was calculated from 40 successive determinations for 3 consecutive days, using 3 samples with different concentrations, obtaining the following results:

	Sample 1	Sample 2	Sample 3
<b>Concentration Promedio (<math>\mu\text{g/dL}</math>)</b>	80,48	232,46	30,46
<b>Standard Deviation (<math>\mu\text{g/dL}</math>)</b>	6,57	8,43	2,22
<b>Coefficiente de Variación (%)</b>	8,16	3,63	7,28

#### Sensitivity

The sensitivity was calculated from 40 determinations from a sample free of iron. The average found was  $1,48 \mu\text{g/dL}$  with standard deviation of  $0,51 \mu\text{g/dL}$ . The sensitivity, which indicates the method detection limit, corresponds the average plus 3 times the standard deviation and is equal to  $2,99 \mu\text{g/dL}$ .

#### Linearity

The reaction is linear up to  $1500 \mu\text{g/dL}$ . For larger values than  $1500 \mu\text{g/dL}$ , dilute the serum with distilled water, repeat dosage and multiply the result by a factor of dilution.

## DIAGNOSTIC SIGNIFICANCE

Iron is the most important transition metal and abundant in the human body. It is essential to the synthesis of hemoglobin and myoglobin, besides being present in several biochemical reactions of metabolism.

The Serum Iron is elevated in sideroblastic anemia, hemolytic anemias, hereditary anemias (Thalassemia Major), acute hepatitis, chronic liver diseases, hemochromatosis and massive blood transfusions.

Low Serum Iron levels are found in chronic infection diseases, neoplasms, parasites, menstrual period, large bleeding, pregnancy and dietary deficiencies.

## BIBLIOGRAPHIC REFERENCES

- 1 - PETERS T., et coll.: J. Lab and Clin Med. 48:280,1956.
- 2 - HENRY, R. J., Cannon. D. C. and Winkleman, J. W., eds.: Clinical Chemistry: Principles and Technics, Haper and Row Publishers, 1974.
- 3 - ZAK, B.: Clin Chim. Acta 3: 328, 1958.
- 4 - BRITTENHAM, G.: Clin Chim. Acta. 91: 203, 1979.
- 5 - TONKS, D. B.: Chim. Acta 2: 393, 1983.
- 6 - BURTIS, C.A., ASHWOOD, E.R., Tietz Textbook of Clinical Chemistry, 2ª Edição, Philadelphia: W.B. Saunders, 1986:2175-2211.
- 7- WHO. Use of anticoagulants in diagnostic laboratory investigations and stability of blood, plasma and serum samples. WHO/DIL/LAB/99.1 rev. 2, 2002:31.

## QUALITY ASSURANCE

Before being released for consumption, all **Bioclin** reagents are tested by the Department of Quality Control. The quality of reagents is assured until expiration date stated on the presentation packaging, when stored and transported under appropriate conditions.

## QUIBASA QUÍMICA BÁSICA Ltda

Rua Teles de Menezes, 92 - Santa Branca  
CEP 31565-130 - Belo Horizonte - MG - Brasil  
Phone: +55 (31) 3439.5454 - Fax: +55 (31) 3439.5455  
E-mail: [bioclin@bioclin.com.br](mailto:bioclin@bioclin.com.br)  
CNPJ: 19.400.787/0001-07 - Made in Brazil

## CUSTOMER SERVICE

Customer Advisory Service  
Phone: 0800 0315454  
E-mail: [sac@bioclin.com.br](mailto:sac@bioclin.com.br)

ANVISA registration for Fe Automated kit: 10269360316

Review: February/2019

## UNIVERSAL SYMBOLOLOGY

	CATALOG NUMBER		MANUFACTURED BY
	BATCH CODE		CONTROL
	DATE OF MANUFACTURE		POSITIVE CONTROL
	USED BY (last day of month)		NEGATIVE CONTROL
	TEMPERATURE LIMITATION (store at)		BIOLOGICAL RISK
	CONTAINS SUFFICIENT FOR <N> TESTS		INFLAMMABLE
	CONSULT INSTRUCTIONS FOR USE		CORROSIVE
	IN VITRO DIAGNOSTIC DEVICE		POISON
	EUROPEAN AUTHORIZED REPRESENTATIVE		CE MARK
	KEEP AWAY FROM SUNLIGHT		DO NOT USE IF PACKAGE IS DAMAGED