FINALIDADE

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- 1 - Hirsh J , Deykin D, Poller L: Therapeutic Range For Oral Anticoagulant Therapy. Chest 89/12 (Suppl) 115-155, 1986.
- 2 - International Committee For Standardization In Haematology And International Committee On Trombosis And Haemostasis. Amer. J. Clin. Path . 88:779 1985.
- 3 - National Committee For Clinical Laboratory Standards: Collecion, Transport And Processing Of Blood Specimens For Coagulation Testing And Performance Of Coagulation Assays, 1991. Nccls.
- 4 - Douketis Jd, Lane A , Milne J , Ginsberg Js. Trombosis Research , 1998 ; 92 : 11-17.
- 5 - QUIBASA: Dados do Departamento de Pesquisa e Desenvolvimento.

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USAGE INSTRUCTIONS

FUNCTION

Method for determination of manual or automated of Prothrombin Time in citrated plasma. For *in vitro* diagnostic use only.

PRINCIPLE OF ACTION

Methodology: Coagulometry

Consists of measuring the Clotting Time of plasma after addition a source of Thromboplastin (factor III, tissue factor) and Calcium. The factor VII is then activated to form a complex (complex FT-FVIIa) that will activate factors IX and factor Xa XO with Phospholipids factor tissue, Calcium and factor Va form the Prothrombin activator complex which transforms Prothrombin (factor II) into Thrombin (factor IIa). Thrombin acts on Fibrinogen, transforming it into Fibrin. This test assesses the factors of Prothrombin Complex (factors II, V, VII and X).

REAGENT

Number 1 - Clot Former - Store between 2 and 8°C. Contains: Rabbit brain extract and preservative.
Presentation ready for use.

PRESENTATION

Presentation	Reagent 1
1	10 x 5 mL
2	20 x 2 mL
3	10 x 2 mL
4	20 x 5 mL

EQUIPMENTS AND OPERATIONAL INPUTS

Log-log paper, pipette and tips, tubes, coagulometer or stopwatch, Biocontrol N and P Coagulation Bioclin. Can be found in stores specialized in articles for Laboratories of Clinical Analysis.

TRANSPORTATION AND STORAGE CONDITIONS

The storage temperature should be 2 to 8°C. Transportation, at temperatures between 15 and 30°C, should not exceed 72 (seventy-two) hours. Protect from light and avoid moisture.
Do not freeze.

SPECIAL CARE

- 1- For professional *in vitro* diagnostic use only.
- 2- Follow strictly the proposed methodology to obtain results accurate.
- 3- Handle with care reagents because they contain Sodium Azide, irritating to skin and mucous membranes.
- 4- We recommend applying the local, state and federal rules for environmental protection, so that disposal of reagents and biological material can be made in accordance with current legislation.
- 5- To obtain information related to biosafety or in case of accidents with the product, consult the MSDS (Safety Data Sheet Safety of Chemicals) available on the website www.bioclin.com.br or upon request by the SAC (Advisory Service Customer) of Quibasa.
- 6- Do not use the product in case of damaged packaging.
- 7- It is essential that the instruments and equipments used are properly calibrated and subjected to periodic maintenance.

SAMPLES

Material: Plasma

Anticoagulant: Use the kit as anticoagulant Citrate Bioclin A001 as written in usage instructions of the same, or 3,8% Sodium Citrate (0,130 M) at a ratio of 9 parts blood to 1 part anticoagulant (Example: 4,5 mL blood + 0,5 mL of anticoagulant).

Collection: Blood should be obtained by venipuncture, avoiding hemolysis, prolonged tourniquet, blistering and aspiration tissue fluid.

Preparation: Blood should be mixed with the anticoagulant soon after collection. Centrifuge immediately at 3000 rpm for 15 minutes. Remove the plasma without cellular pipette red or yellow layer. The samples should be tested in less than three hours. If the test can not be done in this period the plasma should be frozen for at maximum 1 week at -20°C.

PROCESS DESCRIPTION

TECHNIQUE

Bioclin recommends, as control serum, Biocontrol N and P Coagulation Bioclin kits.

- 1- Preheat the Reagent N° 1 (Clot Former) at 37°C for 4 to 5 minutes (Heat only what is necessary for the test).
- 2- Homogenize the reagent before use.
- 3- In a clean tube, pipet 100 µL of the plasma to be measured (Controls or Samples) and incubate at 37 ° C for 2 to 3 minutes (in a water bath or thermal blocks).
- 4- Add 200 µL of reagent N° 1 (Clot Former), previously heat, to the tube containing the plasma and trigger the timer.
- 5- Leave the tube on the heating wand slowly.
- 6- Remove the tube before the expected coagulation time (approximately 9 seconds), observe the formation of the clot. Immediately stop the timer by observing the formation of the clot and recording the time.

Note: After opened, reagent N° 1 is stable for 14 days at 2 to 8°C.

CALCULATIONS AND RESULTS

The results of Prothrombin Time may be expressed as:

Quick Time: In seconds.

Percentage of activity: Related to a normal plasma (considered 100% activity), and for this you must draw a curve activity, using a normal plasma pool and freshly harvested and diluted with saline (0,9%).

Curve

Dilution	1:1	1:2	1:3	1:4	1:8	1:10
Pool (mL)	0,2	0,2	0,2	0,2	0,2	0,2
Physiologic Solution (mL)	---	0,2	0,4	0,6	1,4	1,8
Activity	100%	50%	33%	25%	12,5%	10%

In a log-log paper, plot the graph by putting the seconds value on the “y” axis and the percentage value on the “x” axis.
This procedure should be repeated on every batch.

INR (International Normalized Ratio)

To obtain INR must be related to the prothrombin time ratio the patient by the time of normal plasma with the ISI (Sensitivity Index International), as shown below. The value of ISI is sent with the kit.

INR = (PT patient / PT normal) ^{ISI}

Patient PT: 13,5 sec

Normal PT (reference): 12 sec

ISI: 1,20

Example:

INR = (13,5 / 12) ^{1,20} = 1,15

INTERFERENCES

Excessive hemolysis and lipemia. Use of plasma with Heparin and EDTA.

The PT can be increased by substances such as Anticoagulants Heparin and Warfarin, Oral Contraceptives, Ethanol, Tetracycline, Corticosteroids and EDTA.

The reduction of PT is observed in patients taking: Vitamin K, Anti-Histamines, Caffeine and Phenobarbital.

INTERNAL QUALITY CONTROL

The Clinical Laboratory must have an internal quality control, where all procedures, rules, limits and tolerance to variations be clearly established. It is important to mention that all measurement systems present a analytical variety, and it must be monitor by the laboratory. Therefore, it is recommendable the use of controls, allowing the precision and accuracy of the dosages.

TRACEABILITY

The PT Bioclin kit is traceable to the reference material NIBSC RBT/05 (International Standard for Thromboplastin - WHO International Standard).

REFERENCE VALUES

Prothrombin time (quick) - 10 to 14 seconds

Activity (%) - 70 to 100%

INR - 1,0 to 1,08 in healthy people

- 2,0 to 3,5 in patients taking oral anticoagulants

Each laboratory should establish its reference values, and the results vary with respect to the reagent used. The results should be expressed in INR to normalize the variation of results between laboratories.

ISI values are determined by comparison with a reference material for thromboplastin primary. As low as the ISI is, the more sensitive is the reagent.

The results provided by this kit must be interpreted by the medical professional responsible, not being the only criterion for the determination of diagnosis and/or treatment of the patient.

PRODUCT PERFORMANCE
QUALITY CONTROL

Accuracy

COMPARISON OF METHODS AND SPECIFICITY
METHODOLOGY

PT Bioclin kit was compared with another method for evaluating Prothrombin Time commercially available.

Were performed 07 analysis and the results were evaluated. The obtained linear equation was Y = 1,0804X - 1,2695 and correlation coefficient 0,9954. With these results, we can conclude that the kit shows good methodological specificity.

Specificity

The specificity of the kit is provided by the addition of an exclusive source thromboplastin, specific activator of factor VII, which eliminates the possibility of direct activation of the cascade of other factors coagulation.