TRANSAMINASE AST (TGO) CINÉTICA

[REF] **K048**

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FINALIDADE

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USAGE INSTRUCTIONS

FUNCTION

Test for determination of Aspartate Amino Transferase (AST or TGO) in biological sample (serum or plasma), through kinetic test. Only for *in vitro* diagnostic use.

PRINCIPLE OF ACTION

Methodology: Kinetic (UV)

AST catalyzes the transfer of Amine groups from Aspartate to Alfa - Ketoglutarate, leading to the formation of Glutamate and Oxalacetate. Oxalacetate in the presence of MDH reacts with NADH, reducing to Malate and NADH oxidizes to NAD +. The oxidation rate is proportional to the activity of AST in the sample.

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Oxaloacetato + NADH + H+ **←MDH→** L - Malato + NAD+

REAGENTS

Reagent Nº 1 - Substrate - Store between 2 e 8 °C. Contain Buffer < 200 mmol/L, Sodium Aspartate < 450 mmol/L, D-Lactate Dehydrogenase < 5 KU, Malate Dehydrogenase < 5 KU, chelator, surfactant and preservative.

Reagent Nº 2 - Coenzyme - Store between 2 e 8 °C. Contém: Tris Buffer < 300 mmol/L, Alpha Ketoglutaric Acid < 180 mmol/L, NADH < 5 mmol/L, surfactant and preservative.

PRESENTATION

Presentation	Reagent Nº 1	Reagent Nº 2
1	2 x 40 mL	2 x 10 mL
2	4 x 40 mL	4 x10 mL

EQUIPMENTS AND OPERATIONAL INPUTS

Thermostatic spectrophotometer, pipettes and tips, test tubes, clock or stopwatch, Biocal, Biocontrol N e Biocontrol P Bioclin. These items are found in the specialized market for articles for Clinical Analysis Laboratories.

TRANSPORTATION AND STORAGE CONDITIONS

The storage temperature should be between 2 to 8 °C. The transport, at temperatures up to 30 °C, should not exceed 5 days. Protect from light and avoid moisture. **Do not freeze**.

SPECIAL CARE

1- For professional *in vitro* diagnostic use only.

2- Strictly follow the methodology proposed to obtain exact results.

3- Water used in material cleaning must be recent and free of contaminants.

4- Saturated deionizing columns release alkaline water, various ions and oxidizing and reducing agents, which can significantly alter the results.

5-Visible hemolysis can cause slight variations in results.

6- We recommend applying local, state and federal environmental protection standards so that the disposal of reagents and biological material is carried out in accordance with current legislation.

7- To obtain information related to biosafety or in case of accidents with the product, consult the MSDS (Safety Information Sheet for Chemical Products) available on the website www.bioclin.com.br or upon request by the Customer Service Customer) of Quibasa

8-Do not use the product in case of damage to the packaging.

9- It is essential that the instruments and equipment used are properly calibrated and subjected to periodic maintenance.

10-Reagents No. 1 and 2 contain sodium azide and should be handled with care.

11- - It is important, for the good performance of the test, a strict control of time, temperature and pH.

12- Lipemic and icteric samples increase the absorbance by 340 nm. In this case, dilute the sample 1: 2 with physiological saline multiply the result obtained by 2.

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13- Samples stored without centrifugation for long periods may have their pyruvate concentration increased, which may alter the results. See more information in Interferents.

14- Absorbance readings below 0.800 of the Working Reagent indicate loss of the same. In this case, do not use the reagent.

SAMPLES

Serum or Plasma: collected with EDTA or Heparin, obtained free from hemolysis. The enzyme is stable for 3 days at 2 to 8 °C and 3 months at -20 °C.

PROCESS DESCRIPTION PREPARATION OF WORKING REAGENT
Mix 4 parts of Reagent Nº 1 with 1 part of Reagent Nº 2. Working Reagent is stable during 72 hours between 15 and 30 °C and 14 days between 2 and 8 °C.

The stability of the Transaminasa AST (TGO) product Kinetics after opening (In use / On board - Product installed in equipment) is 30 days. This stability can vary depending on the condition of the problem. Therefore, it is suggested to monitor the performance of the product using control numbers.

REACTION CONDITIONS

It is indispensable to use a thermostated cuvette at 37 °C, 1 cm optical path and reading at 340 nm (334 - 365 nm).

TECHNIQUE

Bioclin recommends using the Biocal kit as a calibrator and the Biocontrol N and P Bioclin kits as the control serum.

Add 100 µL of Sample to 1.0 mL of the Working Reagent, mix and transfer to a thermostated cell at 37 °C and wait 1 minute. Take the initial reading, simultaneously triggering the stopwatch. Repeat the readings after 1, 2 and 3 minutes. Calculate the average of the absorbance differences per minute (ΔAbs. / Min.) And use it to calculate the result.

DESCRIPTION OF THE CALCULATIONS

AST - Results expressed in U/L

340 nm = ΔAbs./min. x 1746

334 nm = ΔAbs./min. x 1780

365 nm = ΔAbs/min. x 3235

PROCESS LIMITATIONS

The specifications below refer to semi-automatic equipment:

The kinetic method is based on molar absorptivity, for this reason, the readings must be performed on a spectrophotometer that fulfills the following conditions:

- Wavelength 340 nm
- Semi-trajectory of the 10 nm passband
- Spurious light less than 0.5%
- Thermostatized 1cm bucket

INTERFERENCE

No interference was observed by Bilirubin until 19 mg/dL, Hemoglobin until 45 mg/dL, Triglycerides until 650 mg/dL and Pyruvate until 0.2 mmol/L.

INTERNAL QUALITY CONTROL

The Clinical Laboratory must have an internal quality control, where all procedures, rules, limits and tolerance to variations be clearly established. It is important to mention that all measurement systems present a analytical variety, and it must be monitor by the laboratory. Therefore, it is recommendable the use of controls, allowing the precision and curacy of the dosages.

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- 1 - The Committee on Enzymes of the Scandinavian Society for Clinical Chemistry and Clinical Physiology Scand J. Clin Lab Invest., 1974, 33, 291 - 306.
- 2 - BERGMEYER, Bowers and cols., Clin. Chim., Acta., 1.976, 70, 19 - 42, 1977, 21 - 22.
- 3 - BERGMEYER, HV.; SCHEIBE, P.; WAHLEFELD, A.W., Clin., Chem., 1.978, 24, 58 - 73.
- 4 - Expert Panel on Enzymes of the International Federation of Clinical Chemistry: Part 3. Revised IFCC Method for Aspartate Aminotransferase, Clin.,Chem., 1.978, 24, 720.
- 5 - Scandinavian Committee on Enzymes Scand J. Clin Lab Invest, 1.981, 41, 107 - 116.
- 6 - BURTIS; CARL, A.; ASHWOOD; EDWARD, R., Clin. Chem., Tietz TEXT BOOK of; 2a ed., 1.986, 788 - 797.
- 7 - PESCE, A., J.; KAPLAN, L., A., Methods in Clin. Chem., 1.987.
- 8 - SOLDIN, S.J., BRUGNARA, C., WONG, E.C.: Pediatric Reference Intervals, 5.ed. Washington: AACC Press, 2005.p.3-4.
- 9 - WHO. Use of anticoagulants in diagnostic laboratory investigations and stability of blood, plasma and serum samples. WHO/DIL/LAB/99.1 rev. 2, 2002:31
- 10 - ELLIS S. Benson M.D. & D. B. Tonks PH.D. (1963) Laboratory medicine, Postgraduate Medicine, 34:4, A-58-A-70, DOI: 10.1080/00325481.1963.11694890
- 11 - QUIBASA: Dados do Departamento de Pesquisa e Desenvolvimento.

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- 1 - The Committee on Enzymes of the Scandinavian Society for Clinical Chemistry and Clinical Physiology Scand J. Clin Lab Invest., 1974, 33, 291 - 306.
- 2 - BERGMEYER, Bowers and cols., Clin. Chim., Acta., 1.976, 70, 19 - 42, 1977, 21 - 22.
- 3 - BERGMEYER, HV.; SCHEIBE, P.; WAHLEFELD, A.W., Clin., Chem., 1.978, 24, 58 - 73.
- 4 - Expert Panel on Enzymes of the International Federation of Clinical Chemistry: Part 3. Revised IFCC Method for Aspartate Aminotransferase, Clin.,Chem., 1.978, 24, 720.
- 5 - Scandinavian Committee on Enzymes Scand J. Clin Lab Invest, 1.981, 41, 107 - 116.
- 6 - BURTIS; CARL, A.; ASHWOOD; EDWARD, R., Clin. Chem., Tietz TEXT BOOK of; 2a ed., 1.986, 788 - 797.
- 7 - PESCE, A., J.; KAPLAN, L., A., Methods in Clin. Chem., 1.987.
- 8 - SOLDIN, S.J., BRUGNARA, C., WONG, E.C.: Pediatric Reference Intervals, 5.ed. Washington: AACC Press, 2005.p.3-4.
- 9 - WHO. Use of anticoagulants in diagnostic laboratory investigations and stability of blood, plasma and serum samples. WHO/DIL/LAB/99.1 rev. 2, 2002:31
- 10 - ELLIS S. Benson M.D. & D. B. Tonks PH.D. (1963) Laboratory medicine, Postgraduate Medicine, 34:4, A-58-A-70, DOI: 10.1080/00325481.1963.11694890
- 11 - QUIBASA: Datos del Departamento de Investigación y Desarrollo.

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PELIGRO

Sensitivity

Sensitivity was calculated from 40 determinations of a sample free from Aspartate aminotransferase. The average found was 2.10 U/L with a standard deviation of 0.585 U/L. The sensitivity, which indicates the detection limit of the method, corresponds to 3 times the standard deviation, and is equal to 1.756.

Linearity

The reaction is linear up to a concentration of 400 U/L. For samples with values above 400 U/L, it is recommended to dilute the sample with 0.85% Sodium Chloride, repeat the dosage and multiply the result obtained by the dilution factor. For example, if the sample is diluted 1:10 and the result obtained is 80 U/L, multiply by 10 (80 x 10) U/L and consider the activity value of 800 U/L as the final result.

DIAGNOSTIC SIGNIFICANCE

The increase in the activity of the enzymes Aspartate AminoTransferase - AST (located in the cytomitochondrial region) reflects changes in several tissues. This enzyme is found in high concentration in the heart, liver, skeletal muscle, kidneys and pancreas. Its plasma activity increases after 6 to 8 hours after myocardial infarction, reaching a peak in 24 to 48 hours after the onset. Considerable increases occur in viral, toxic hepatitis, liver necrotic diseases - 3 to 50 or 100 times the reference values (VR), mononucleoses (20 times the VR), intrahepatic cholestasis (20 times the VR) and muscular dystrophies (8 times the VR). In chronic liver diseases associated with cell necrosis, due to the increase in AST, inversion of the ALT (TGP) / AST ratio may occur.

BIBLIOGRAPHIC REFERENCES

- 1 - The Committee on Enzymes of the Scandinavian Society for Clinical Chemistry and Clinical Physiology Scand J. Clin Lab Invest., 1974, 33, 291 - 306.
- 2 - BERGMEYER, Bowers and cols., Clin. Chim., Acta., 1.976, 70, 19 - 42, 1977, 21 - 22.
- 3 - BERGMEYER, HV.; SCHEIBE, P.; WAHLEFELD, A.W., Clin., Chem., 1.978, 24, 58 - 73.
- 4 - Expert Panel on Enzymes of the International Federation of Clinical Chemistry: Part 3. Revised IFCC Method for Aspartate Aminotransferase, Clin.,Chem., 1.978, 24, 720.
- 5 - Scandinavian Committee on Enzymes Scand J. Clin Lab Invest, 1.981, 41, 107 - 116.
- 6 - BURTIS; CARL, A.; ASHWOOD; EDWARD, R., Clin. Chem., Tietz TEXT BOOK of; 2a ed., 1.986, 788 - 797.
- 7 - PESCE, A., J.; KAPLAN, L., A., Methods in Clin. Chem., 1.987.
- 8 - SOLDIN, S.J., BRUGNARA, C., WONG, E.C.: Pediatric Reference Intervals, 5.ed. Washington: AACC Press, 2005.p.3-4.
- 9 - WHO. Use of anticoagulants in diagnostic laboratory investigations and stability of blood, plasma and serum samples. WHO/DIL/LAB/99.1 rev. 2,2002:31
- 10 - ELLIS S. Benson M.D. & D. B. Tonks PH.D. (1963) Laboratory medicine, Postgraduate Medicine, 34:4, A-58-A-70, DOI: 10.1080/00325481.1963.11694890
- 11- QUIBASA: Data from the Research and Development Department.

QUALITY ASSURANCE

Before being released for consumption, all Bioclin reagents are tested by the Department of Quality Control. The quality of reagents is assured until expiration date stated on the presentation packaging, when stored and transported under appropriate conditions.

QUIBASA QUÍMICA BÁSICA Ltda
Rua Teles de Menezes, 92 - Santa Branca
CEP 31565-130 - Belo Horizonte - MG - Brasil
Phone.: +55 (31) 3439.5454
E-mail: bioclin@bioclin.com.br
CNPJ: 19.400.787/0001-07 - Made in Brazil

EC

REP

OBELIS S.A.

Bd. Général Wahis, 53
1030 Brussels, Belgium

CUSTOMER SERVICE

Customer Advisory Service
Phone: 0800 0315454
E-mail: customerservice@bioclin.com.br

ANVISA registration for Transaminase AST Kinetic kit: 10269360119

Review: July/2021

UNIVERSAL SYMBOLOGY

CATALOG NUMBER

MANUFACTURED BY

BATCH CODE

CONTROL

DATE OF MANUFACTURE

POSITIVE CONTROL

USED BY
(last day of month)

NEGATIVE CONTROL

TEMPERATURE LIMITATION
(store at)

BIOLOGICAL RISK

CONTAINS SUFFICIENT
FOR <N> TESTS

INFLAMMABLE

CONSULT INSTRUCTIONS
FOR USE

CORROSIVE

IN VITRO DIAGNOSTIC DEVICE

POISON

CAUTION

DANGER