FLUORETO

REF A006

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FINALIDADE

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USAGE INSTRUCTIONS

FUNCTION

Anticoagulant solution glucolysis inhibitor. For *in vitro* diagnostic use only.

PRINCIPLE OF ACTION

Anticoagulant Fluoride is formed by a solution of Fluoride and EDTA. This mixture added to whole blood, at the time of collection, prevents glucolysis. The EDTA acts by blocking the ionized calcium, forming an insoluble complex of Ca-EDTA and fluoride inhibits glucolysis, allowing glucose to remain stable for hours between 15 and 30°C.

REAGENTS

Fluoride Anticoagulant - Store between 15 and 30°C. Contains: EDTA < 500 mmol/L, Potassium Fluoride < 6 mol/L and dye.

PRESENTATION

Presentation	Volume
1	20 mL
2	250 mL

EQUIPMENTS AND OPERATIONAL INPUTS

Test tubes and other materials needed to collect blood. They can be found at markets specialized on Laboratories of Clinical Analysis.

TRANSPORTATION AND STORAGE CONDITIONS

The storage and transportation temperatures should be between 15 to 30°C. Protect from light and avoid moisture.

SPECIAL CARE

- 1- For *in vitro* diagnostic use only.
- 2- Water used in material cleaning must be recent and free of contaminants.
- 3- Saturated deionizer columns release alkaline water, many ions, oxidizing agents and reducers that may alter the results significantly.
- 4- We recommend applying the local, state and federal rules for environmental protection, so that disposal of reagents and biological material can be made in accordance with current legislation.
- 5- To obtain information related to biosafety or in case of accidents with the product, consult the MSDS (Material Safety Data Sheet) available on the website www.bioclin.com.br or upon request by the SAC (Advisory Service Customer) of Quibasa.
- 6- Do not use the product in case of damaged packaging.
- 7- It is essential that the instruments and equipments used are properly calibrated and subjected to periodic maintenance.

TECHNIQUE

Use 1 drop of anticoagulant for each 3 mL of blood. Smoothly homogenize through inversion. Separate plasma by centrifuging (3.000 rpm), for 5 minutes. Transfer supernatant to a test tube and maintain plasma under refrigeration up to the moment of usage.

PROCEDURE LIMITATIONS

The volume ratio of anticoagulant/blood must be precise, to avoid signs of hemolysis.

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QUALITY ASSURANCE

Before being released for consumption, all **Bioclin** reagents are tested by the Department of Quality Control. The quality of reagents is assured until expiration date stated on the presentation packaging, when stored and transported under appropriate conditions.

QUIBASA QUÍMICA BÁSICA Ltda

Rua Teles de Menezes, 92 - Santa Branca

CEP 31565-130 - Belo Horizonte - MG - Brasil

Phone.: +55 31 3439.5454 | E-mail: bioclin@bioclin.com.br

CNPJ: 19.400.787/0001-07 - Made in Brazil

CUSTOMER SERVICE

Customer Advisory Service

Phone: 0800 0315454 | E-mail: sac@bioclin.com.br

ANVISA registration for Fluoride kit: 10269360115

Review: May/2022

UNIVERSAL SYMBOLOGY

CATALOG NUMBER

LOT NUMBER

MANUFACTURING DATE

VALIDITY DATE
(last day of the month)

TEMPERATURE LIMIT
(store)

CONTENT IS SUFFICIENT
FOR <N> TEST

SEE INSTRUCTIONS
FOR USE

IN VITRO DIAGNOSTIC
PRODUCT

KEEP AWAY
FROM SUNLIGHT

DO NOT REUSE

CAUTION

MADE BY

CONTROL

POSITIVE CONTROL

NEGATIVE CONTROL

BIOLOGICAL RISK

FLAMMABLE

CORROSIVE

TOXIC

DO NOT USE IF
PACKAGE IS
DAMAGED

PRODUCT
STERILIZED

DANGER