Fe AUTOMAÇÃO

REF K308

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FINALIDADE

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- 1 - Peters T. et al: J. Lab and Clin Med. 48:280,1956.
- 2 - Henry RJ, Cannon DC and Winkelman JW, eds.: Clinical Chemistry: Principles and Technics, Haper and Row Publishers, 1974.
- 3 - Zak B.: Clin Chim. Acta 3: 328, 1958.
- 4 - Brittenham G.: Clin Chim. Acta. 91: 203, 1979.
- 5 - Tonks DB.: Chim. Acta 2: 393, 1983.
- 6 - Burtis CA, Ashwood ER. Tietz Textbook of Clinical Chemistry, 2ª Edição, Philadelphia: W.B. Saunders, 1986:2175-2211.
- 7 - WHO. Use of anticoagulants in diagnostic laboratory investigations and stability of blood, plasma and serum samples. WHO/DIL/LAB/99.1 rev. 2, 2002:31.
- 8 - Goodwin et al. Direct measurement of serum iron and binding capacity. Clinical Chemistry. Vol.12, N° 2, 1966
- 9 - Stookey LL. Ferrozine – A new spectrophotometric reagent for iron. Analytical Chemistry, Vol. 42, N° 7, June 1970.
- 10 – White JM and Flashka HA. An Automated Procedure, with Use of Ferrozine, for Assay of Serum Iron and Total Iron-Binding Capacity. Clinical. Chemistry. 19/5, 526-528, 1973.
- 11 - Muckenyhalet et al. A red carpet for Iron Metabolism. Cell. 26; 168(3): 344–361, January 2017.
- 12 - EClinPalth. Cornell University College of veterinary Medicine. 2013.
- 13 - QUIBASA: Dados do Departamento de Pesquisa e Desenvolvimento.

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- 1 - Peters T. et al: J. Lab and Clin Med. 48:280,1956.
- 2 - Henry RJ, Cannon DC and Winkleman JW, eds.: Clinical Chemistry: Principles and Technics, Haper and Row Publishers, 1974.
- 3 - Zak B.: Clin Chim. Acta 3: 328, 1958.
- 4 - Brittenham G.: Clin Chim. Acta. 91: 203, 1979.
- 5 - Tonks DB.: Chim. Acta 2: 393, 1983.
- 6 - Burtis CA, Ashwood ER. Tietz Textbook of Clinical Chemistry, 2ª Edição, Philadelphia: W.B. Saunders, 1986:2175-2211.
- 7 - WHO. Use of anticoagulants in diagnostic laboratory investigations and stability of blood, plasma and serum samples. WHO/DIL/LAB/99.1 rev. 2, 2002:31.
- 8 - Goodwin et al. Direct measurement of serum iron and binding capacity. Clinical Chemistry. Vol.12, N° 2, 1966
- 9 - Stookey LL. Ferrozine – A new spectrophotometric reagent for iron. Analytical Chemistry, Vol. 42, N° 7, June 1970.
- 10 – White JM and Flashka HA. An Automated Procedure, with Use of Ferrozine, for Assay of Serum Iron and Total Iron-Binding Capacity. Clinical. Chemistry. 19/5, 526-528, 1973.
- 11 - Muckenyhalet et al. A red carpet for Iron Metabolism. Cell. 26; 168(3): 344–361, January 2017.
- 12 - ECLinPahtn. Cornell University College of veterinary Medicine. 2013.
- 13 - QUIBASA: Dados do Departamento de Pesquisa e Desenvolvimento.

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Fe AUTOMATEDREF **K308****INSTRUCTIONS FOR USE****FUNCTION**

Method for the quantitative determination of Iron serum and plasma (Heparin) samples. Colorimetric test, for *in vitro* diagnostic use only.

PRINCIPLE OF ACTION

Methodology: Modified Goodwin/Ferrozine
Iron is released from Transferrin in an acid environment and lowered to its ferrous state by the action of Hydroxylamine. Subsequently, reacts with Ferrozine leading to the formation of a violet colored complex.

REAGENTS

Number 1 - Reduction Buffer - Store between 2 and 8 °C. Contains: Sodium Acetate < 900 mmol/L, Hydroxylamine < 800 mmol/L, surfactants and stabilizer.

Number 2 - Color Reagent - Store between 2 and 8 °C. Contains: Ferrozine < 25 mmol/L, Hydroxylamine < 20 mmol/L and stabilizers.

Number 3 - Standard - Store between 2 and 8 °C. Contains: Iron 200 µg/dL, stabilizer and preservative.

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EQUIPMENTS AND OPERATIONAL INPUTS

Biochemistry automated equipment or semi-automated, water bath, pipettes, test tubes, Biocontrol N Bioclin and Biocontrol P Bioclin. These items are found at markets specialized on Laboratories of Clinical Analysis.

TRANSPORTATION AND STORAGE CONDITIONS

Store temperatures must be between 2 and 8 °C. The transport at temperatures between 15 and 30 °C should not exceed 5 days. Keep away from light and avoid humidity. **Do not freeze.**

SPECIAL CARE**1- For *in vitro* diagnostic use only.**

2- Strictly follow the methodology proposed to obtain exact results.

3- Water used in material cleaning must be recent and free of contaminants.

4- Saturated deionizer columns release alkaline water, many ions, oxidizing agents and reducers that may alter the results significantly.

5- Hemolysis, even if discrete, can interfere with dosage.

6- Every material used for analyses must be previously immersed in a recent HCL (7%) solution for 30 minutes. Rinse it with deionized water.

7- We recommend applying the local, state and federal rules for environmental protection, so that disposal of reagents and biological material can be made in accordance with current legislation.

8- To obtain information related to biosafety or in case of accidents with the product, consult the SDS (Safety Data Sheet) available on the website www.bioclin.com.br or upon request by the SAC

(Customer Advisory Service) of Quibasa.

9- Do not use the product in case of damaged packaging.

10- It is essential that the instruments and equipments used are properly calibrated and subjected to periodic maintenance.

SAMPLES

Serum and plasma (Heparin) obtained free of hemolysis, stable for 21 days between 2 and 8 °C and 1 year at -20 °C. ⁷

PROCESS DESCRIPTION

The calibration stability of the Fe Automated kit installed on refrigerated equipment is up to 19 days. This stability may vary depending on the conditions of the test, equipment and environment. Therefore, it is suggested to follow the product performance using control serum.

TECHNIQUE

Bioclin recommends, as control serum, Biocontrol N Bioclin and Biocontrol P Bioclin Kit.

Mark 3 test tubes with letters B (Blank), P (Standard), A (Sample) and proceed as follows:

	Blank	Standard	Sample
Reagent N°1	0.8 mL	0.8 mL	0.8 mL
Distilled Water	0.1 mL	----	----
Sample	----	----	0.1 mL
Reagent N°3	----	0.1 mL	----

Smoothly homogenize and determine the absorbances of the Sample and Standard at 560 nm (540 - 580 nm), hitting the zero with the distilled water. This will be the absorbance A_1 .

And then add:

	Blank	Standard	Sample
Reagent N°2	0.2 mL	0.2 mL	0.2 mL

Smoothly homogenize and incubate in water-bath at 37°C, per 5 minutes. Determine the absorbances of Sample and the Standard in 560 nm (540 - 580 nm), matching the zero with the Blank. This will be the absorbance of the sample A_2 .

CALCULATIONS

Due to the dilution, the A_1 reading of the Standard and Sample should be corrected for the final volume, using the factor of 0.82. Thus, $A_{1CORRECTED}$ is obtained.

$$\text{Standard-}A_{1CORRECTED} = \text{Standard } A_1 \times 0.82$$

$$\text{Sample-}A_{1CORRECTED} = \text{Sample } A_1 \times 0.82$$

$$\text{Iron Serum } (\mu\text{g/dL}) = \text{Sample } (A_2 - A_{1CORRECTED}) \times 200$$

$$\text{Standard } (A_2 - A_{1CORRECTED})$$

VETERINARY USE

TO OBTAIN THE INSTRUCTIONS FOR USE IN PRINTED FORMAT, AT NO ADDITIONAL COST, CONTACT CUSTOMER ADVISORY SERVICE:

SAC: +55 (31) 3439 5454 / 0800 031 5454 / sac@bioclin.com.br

The color reaction follows the Lambert Beer Law, it can use the Calibration Factor:

$$\text{Calibration Factor} = \frac{200}{\text{Standard } (A_2 - A_{1CORRECTED})}$$

$$\text{Iron Serum } (\mu\text{g/dL}) = \text{Sample } (A_2 - A_{1CORRECTED}) \times \text{Calibration Factor}$$

The results are expressed in µg/dL.

PROCEDURE LIMITATIONS

The use of water contaminated by Iron increase white color intensity, changing the results.

INTERFERENCE

Triglycerides up to 1000 mg/dL, Bilirubin up to 20 mg/dL, Ascorbic Acid up to 60 mg/dL, and Rheumatoid Factor up to 300 IU/mL do not interfere with the results.

Some drugs can raise Serum Iron levels such as: Chloramphenicol, Estrogens, Ethanol, Methotrexate and Anovulatory drugs. Other drugs may decrease Serum Iron levels such as: Acetylsalicylic Acid (in high doses), Allopurinol, Cortisone and Corticotropin.

Patients undergoing hemodialysis or using anticoagulants (with the exception of Heparin) may have falsely elevated iron results due to the high concentration of fibrinogen. The presence of hemoglobin also produces significantly elevated results.

Heparin anticoagulants do not interfere with the determination of the analyte. Plasma (Heparin) can be used as a sample for the Fe Automation kit.

INTERNAL QUALITY CONTROL

The Clinical Laboratory must have an internal quality control, where all procedures, rules, limits and tolerance to variations be clearly established. It is important to mention that all measurement systems present a analytical variety, and it must be monitor by the laboratory. Therefore, it is recommendable the use of controls, allowing the precision and accuracy of the dosages.

TRACEABILITY

The kit's standard is traceable to the reference material NIST (National Institute of Standards and Technology) SRM 937.

REFERENCE VALUES

The reference values were determined in healthy dogs, cats, cattle and horses, males and females, adults and mixed breed.

Specie	Reference value
Cães	51.72 - 251.9 µg/dL
Gatos	40.54 - 151.8 µg/dL
Bovinos	68 - 313.67 µg/dL
Equinos	25.05 - 251 µg/dL

These values should be used as guidelines and each laboratory should establish its own reference value, according to the population served. The results provided by this kit must be interpreted by the veterinary medical professional, and are not the only criterion for determining the diagnosis and/or treatment of the patient.

To convert the values from µg/dL to mmol/L, multiply by 0.179.

PRODUCT PERFORMANCE

ACCURACY

Comparison of Methods and Methodological Specificity

The Fe Automated kit was compared with other method for determination of Iron dosage commercially available. 42 analyzes were performed and the results were evaluated. The linear equation obtained was $Y = 0.9744X + 1.6825$ and correlation coefficient 0.999. With these results we can conclude that the kit shows good methodological specificity.

PRECISION

Repeatability

The repeatability was calculated from 40 successive determinations, using 3 samples with different concentrations, obtaining the following results:

Repeatability	Sample 1	Sample 2	Sample 3
Concentration Promedio (µg/dL)	72.90	222.73	27.90
Standard Deviation (µg/dL)	1.43	0.93	0.63
Coefficient of Variation (%)	1.96	0.42	2.27

Reproducibility

The reproducibility was calculated from 40 successive determinations for 3 consecutive days, using 3 samples with different concentrations, obtaining the following results:

Reproducibility	Sample 1	Sample 2	Sample 3
Concentration Promedio (µg/dL)	80.48	232.46	30.46
Standard Deviation (µg/dL)	6.57	8.43	2.22
Coefficient of Variation (%)	8.16	3.63	7.28

SENSITIVITY

The sensitivity was calculated from 40 determinations from a sample free of iron. The average found was 1.48 µg/dL with standard deviation of 0.51 µg/dL. The sensitivity, which indicates the method detection limit, corresponds the average plus 3 times the standard deviation and is equal to 2.99 µg/dL.

LINEARITY

The reaction is linear up to 1500 µg/dL. For larger values than 1500 µg/dL, dilute the serum with distilled water, repeat dosage and multiply the result by a factor of dilution.

DIAGNOSTIC SIGNIFICANCE

Iron performs fundamental functions in the body and constitutes an important component of hemoglobin, essential for the transport of oxygen. Factors that affect iron concentrations are absorption, utilization by cells, hemocateresis, and loss

BIBLIOGRAPHIC REFERENCES

- 1 - Peters T. et al: J. Lab and Clin Med. 48:280,1956.
- 2 - Henry RJ, Cannon DC and Winkleman JW, eds.: Clinical Chemistry: Principles and Technics, Haper and Row Publishers, 1974.
- 3 - Zak B.: Clin Chim. Acta 3: 328, 1958.
- 4 - Brittenham G.: Clin Chim. Acta. 91: 203, 1979.
- 5 - Tonks DB.: Chim. Acta 2: 393, 1983.
- 6 - Burtis CA, Ashwood ER. Tietz Textbook of Clinical Chemistry, 2ª Edição, Philadelphia: W.B. Saunders, 1986:2175-2211.
- 7 - WHO. Use of anticoagulants in diagnostic laboratory investigations and stability of blood, plasma and serum samples. WHO/DIL/LAB/99.1 rev. 2, 2002:31.
- 8 - Goodwin et al. Direct measurement of serum iron and binding capacity. Clinical Chemistry. Vol.12, N° 2, 1966
- 9 - Stookey LL. Ferrozine - A new spectrophotometric reagent for iron. Analytical Chemistry, Vol. 42, N° 7, June 1970.
- 10 - White JM and Flashka HA. An Automated Procedure, with Use of Ferrozine, for Assay of Serum Iron and Total Iron-Binding Capacity. Clinical. Chemistry. 19/5, 526-528, 1973.
- 11 - Muckenyhalet et al. A red carpet for Iron Metabolism. Cell. 26; 168(3): 344-361, January 2017.
- 12 - EClinPahth. Cornell University College of veterinary Medicine. 2013.
- 13 - QUIBASA: Dados do Departamento de Pesquisa e Desenvolvimento.

QUALITY ASSURANCE

Before being released for consumption, all **Bioclin** reagents are tested by the Department of Quality Control. The quality of reagents is assured until expiration date stated on the presentation packaging, when stored and transported under appropriate conditions.

QUIBASA QUÍMICA BÁSICA Ltda

Rua Teles de Menezes, 92 - Santa Branca
CEP 31565-130 - Belo Horizonte - MG - Brasil
Phone: +55 31 3439.5454 | E-mail: bioclin@bioclin.com.br
CNPJ: 19.400.787/0001-07 - Made in Brazil

CUSTOMER SERVICE

Customer Advisory Service
Phone: 0800 0315454 | E-mail: sac@bioclin.com.br

Registration Number: Product exempt from registration with the Ministry of Agriculture, Livestock and Supply.

Technical Responsible: Dr. Camila Eckstein (CRMV/ MG 20.611)

Review: June/2025

UNIVERSAL SYMBOLOGY

	CATALOG NUMBER		MADE BY
	LOT NUMBER		CONTROL
	MANUFACTURING DATE		POSITIVE CONTROL
	VALIDITY DATE (last day of the month)		NEGATIVE CONTROL
	TEMPERATURE LIMIT (store)		BIOLOGICAL RISK
	CONTENT IS SUFFICIENT FOR <N> TEST		FLAMMABLE
	SEE INSTRUCTIONS FOR USE		CORROSIVE
	IN VITRO DIAGNOSTIC PRODUCT		TOXIC
	KEEP AWAY FROM SUNLIGHT		DO NOT USE IF PACKAGE IS DAMAGED
	DO NOT REUSE		PRODUCT STERILIZED
	CAUTION		DANGER