MULTICONTROL 2

REF K247-1

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FINALIDADE

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INSTRUCTIONS FOR USE

FUNCTION

Reagent used as a control for turbidimetric tests, for *in vitro* diagnostic use only.

REAGENTS

Number 1 (R1) - Multicontrol 2 - Store between 2 and 8°C. Contains: Serum Proteins and preservative. **Potentially infectious.**

Warning: The concentration of analytes varies each lot. See table that accompanies the product.

PRESENTATION

Reagent Nº 1	1 mL
Reagent Nº 1	2 mL
Reagent Nº 1	3 mL
Reagent Nº 1	4 mL
Reagent Nº 1	5 mL
Reagent Nº 1	10 mL
Reagent Nº 1	15 mL
Reagent Nº 1	20 mL

EQUIPMENTS AND OPERATIONAL INPUTS

Automated biochemical devices and kits. These items are found in the specialized market for articles for Clinical Analysis Laboratories.

TRANSPORTATION AND STORAGE CONDITIONS

The storage temperature should be between 2 to 8°C. The transport at temperatures up to 30°C should not exceed 5 days. Protect from light and avoid moisture. **Do not freeze.**

SPECIAL CARE

- 1- For *in vitro* diagnostic use only.**
- 2- Strictly follow the methodology proposed to obtain exact results.
- 3- Water used in material cleaning must to be recent and free of contaminants.
- 4- Saturated deionizer columns release alkaline water, many ions, oxidizing agents and reducers that may alter the results significantly.
- 5- Handle with care, as the reagent contains Sodium Azide, which is irritant to skin and mucous.
- 6- The reagent has been tested for anti-HIV antibodies, and HBs antigen using state of the art methods and found negative results. The risk of infection cannot be excluded and the reagent should be handled as cautiously as observed for the patient's serum. **Potentially Infectious.**
- 7- We recommend applying the local, state and federal rules for environmental protection, so that disposal of reagents and biological material can be made in accordance with current legislation.
- 8- To obtain information related to biosafety or in case of accidents with the product, consult the MSDS (Material Safety Data Sheet) available on the website www.bioclin.com.br or upon request by the SAC (Customer Advisory Service) of Quibasa.
- 9- Do not use the product in case of damaged packaging.
- 10- It is essential that the instruments and equipments used are properly calibrated and subjected to periodic maintenance.

PROCESS DESCRIPTION

The use of the control must obey the specific technique for the equipment to be used.

The concentration of the analytes varies with each batch. See table accompanying the product.

BIBLIOGRAPHIC REFERENCES

- 1 - KILLINGSWORTH, L. M; SAVORY, J.: Automated Immunochemical Procedures for Measurement of Immunoglobulins IgG, IgA and IgM in human serum, Clin. Chem., 1971; 17, 936.
- 2 - PESCE, A. J.; KAPLAN, L. A.: Methods in Clinical Chemistry C. V. Mosby Company, 1987.
- 3 - HELLSING, K., Profiles in the Biological Fluids, 1973 23: 579.
- 4 - BLOM, M. and HJOME, H., Clinical Chemistry, 1976; 22: 657.
- 5 - HILLS, L. P. and TIFFANY, T. I., Clinical Chemistry, 1980; 26: 1459.
- 6 - QUIBASA: Dados do Departamento de Pesquisa e Desenvolvimento.

QUALITY ASSURANCE

Before being released for consumption, all Bioclin reagents are tested by the Quality Control Department. The quality of the reagents is ensured until the expiration date mentioned on the presentation packaging, provided they are stored and transported under the appropriate conditions.

QUIBASA QUÍMICA BÁSICA Ltda
Rua Teles de Menezes, 92 - Santa Branca
CEP 31565-130 - Belo Horizonte/MG - Brasil
Tel.: +55 31 3439 5454 | E-mail: bioclin@bioclin.com.br
CNPJ: 19.400.787/0001-07 - Indústria Brasileira

CUSTOMER SERVICE

Customer Advisory Service
Phone: +55 31 3439 5454 | E-mail: customerservice@bioclin.com.br

ANVISA registration for the Multicontrol 2 kit: 10269360340

Review: November/2023

UNIVERSAL SYMBOLOGY

CATALOG NUMBER

LOT NUMBER

MANUFACTURING DATE

VALIDITY DATE
(last day of the month)

TEMPERATURE LIMIT
(store)

CONTENT IS SUFFICIENT
FOR <N> TEST

SEE INSTRUCTIONS
FOR USE

IN VITRO DIAGNOSTIC
PRODUCT

KEEP AWAY
FROM SUNLIGHT

DO NOT REUSE

CAUTION

MADE BY

CONTROL

POSITIVE CONTROL

NEGATIVE CONTROL

BIOLOGICAL RISK

FLAMMABLE

CORROSIVE

TOXIC

DO NOT USE IF
PACKAGE IS
DAMAGED

PRODUCT
STERILIZED

ANGER