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CLEANSING SOLUTION FOR
AUTOMATED ANALYZER

TO OBTAIN THE INSTRUCTIONS FOR USE IN PRINTED FORMAT,
AT NO ADDITIONAL COST, CONTACT CUSTOMER ADVISORY SERVICE:

SAC: +55 (31) 3439 5454 / 0800 031 5454 / sac@bioclin.com.br

REF L006

USAGE INSTRUCTIONS

FUNCTION

Cleansing solution for automated biochemical devices.

Suitable for:

Automated biochemical devices.

APPLICATION

Cleansing works by removing fat, protein and other products are deposited on the inner walls of the needles of the device.

Internal cleaning of needles provides better results accuracy and reproducibility, eliminating problems of internal bubbles and obstruction.

SOLUTION

Cleansing Solution for Automated Analyzer - Store between 15 and 30°C. Contains surfactant and preservative.

PRESENTATION

Presentation	Volume
1	1 x 200 mL

TRANSPORTATION AND STORAGE CONDITIONS

The storage and transport temperatures must be between 15 to 30°C. Protect from light and avoid moisture.

SPECIAL CARE

1- For *in vitro* diagnostic use only.

2- Strictly follow the methodology proposed to adequate cleansing.

3- Water used in material cleaning must be recent and free of contaminants.

4- Avoid contact with eyes and skin. In case of contact was immediately with water in abundance.

5- Use laboratory gloves when handling this product.

6- Product may be harmful if swallowed or inhaled, if ingested seek medical help.

7- We recommend applying the local, state and federal rules for environmental protection, so that disposal of reagents and biological material can be made in accordance with current legislation.

8- To obtain information related to biosafety or in case of accidents with the product, consult the SDS (Safety Data Sheet) available on the website www.bioclin.com.br or upon request by the SAC (Customer Advisory Service) of Quibasa.

9- Do not use the product in case of damaged packaging.

10- It is essential that the instruments and equipments used are properly calibrated and subjected to periodic maintenance.

PROCESS DESCRIPTION

The Cleansing Solution for Automated Analyzer is ready for use.

Use of this solution must be according to the specifications in manufacturer's device manual.

For further information of this product contact Customer Services.

STABILITY

Three lots of the product Cleansing Solution for Automated Analyzer Bioclin were evaluated within the established expiration date and there was no evidence of any microbiological growth, or loss of physical - chemical characteristics, the product performance remains stable.

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QUALITY ASSURANCE

Before being released for consumption, all **Bioclin** reagents are tested by the Department of Quality Control. The quality of reagents is assured until expiration date stated on the presentation packaging, when stored and transported under appropriate conditions.

QUIBASA QUÍMICA BÁSICA Ltda

Rua Teles de Menezes, 92 - Santa Branca
CEP 31565-130 - Belo Horizonte - MG - Brasil
Phone.: +55 31 3439.5454 | E-mail: bioclin@bioclin.com.br
CNPJ: 19.400.787/0001-07 - Made in Brazil

CUSTOMER SERVICE

Customer Advisory Service
Phone.: 0800 0315454 | E-mail: sac@bioclin.com.br

Product exempt from ANVISA registration, in accordance with current legislation.

Review: April/2025

UNIVERSAL SYMBOLOGY

	CATALOG NUMBER		MADE BY
	LOT NUMBER		CONTROL
	MANUFACTURING DATE		POSITIVE CONTROL
	VALIDITY DATE (last day of the month)		NEGATIVE CONTROL
	TEMPERATURE LIMIT (store)		BIOLOGICAL RISK
	CONTENT IS SUFFICIENT FOR <N> TEST		FLAMMABLE
	SEE INSTRUCTIONS FOR USE		CORROSIVE
	IN VITRO DIAGNOSTIC PRODUCT		TOXIC
	KEEP AWAY FROM SUNLIGHT		DO NOT USE IF PACKAGE IS DAMAGED
	DO NOT REUSE		PRODUCT STERILIZED
	CAUTION		DANGER