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CLEANSING SOLUTION FOR SEMI-AUTOMATED ANALYZER

TO OBTAIN THE INSTRUCTIONS FOR USE IN PRINTED FORMAT, AT NO ADDITIONAL COST, CONTACT CUSTOMER ADVISORY SERVICE:

SAC: +55 (31) 3439 5454 / 0800 031 5454 / sac@bioclin.com.br

REF L001

USAGE INSTRUCTIONS

FUNCTION

Cleansing solution for cuvettes of continuous flow from semi-automated biochemical analyzers.

Suitable for:

Semi-automated biochemical devices.

Application:

Cleansing works by removing fat, protein and other products are deposited on the inner walls of the cuvettes of continuous flow.

Internal cleansing of cuvettes provides better results accuracy and reproducibility, eliminating problems of internal bubbles and obstruction of the optical path.

SOLUTION

Detergent Solution - Store between 15 and 30°C. Contains: 2-Butoxyethanol < 10% and Sodium Hydroxide < 5%.

PRESENTATION

Presentation	Volume
1	1 x 20 mL
2	1 x 50 mL
3	2 x 50 mL
4	4 x 50 mL
5	1 x 500 mL

TRANSPORTATION AND STORAGE CONDITIONS

The storage and transport temperatures should be between 15 to 30°C. Protect from light and avoid moisture.

SPECIAL CARE

- 1- For *in vitro* diagnostic use only.
- 2- Strictly follow the methodology proposed to adequate cleansing.
- 3- Water used in material cleaning must be recent and free of contaminants.
- 4- Avoid contact with eyes and skin. In case of contact was immediately with water in abundance.
- 5- Use laboratory gloves when handling this product.
- 6- Product may be harmful if swallowed or inhaled, if ingested seek medical help.
- 7- We recommend applying the local, state and federal rules for environmental protection, so that disposal of reagents and biological material can be made in accordance with current legislation.

8- To obtain information related to biosafety or in case of accidents with the product, consult the SDS (Safety Data Sheet) available on the website www.bioclin.com.br or upon request by the SAC (Customer Advisory Service) of Quibasa.

9- Do not use the product in case of damaged packaging.

10- It is essential that the instruments and equipments used are properly calibrated and subjected to periodic maintenance.

PROCESS DESCRIPTION

Cleansing Solution is ready for use.

Every day after the end of the routine, vacuum and clean solution then disconnect, leaving the solution in the cuvette over night. Before using the equipment to wash the cuvette with deionized water, washing for at least five times. The methodology used can vary according to the analyzer. For further information, refer to equipment manufacturer.

STABILITY

3 lots of the product Cleansing Solution Bioclin were evaluated during expiration date established and there have not been verified any microbiologica growth, or loss of physical, chemical, product performance remains stable.

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QUALITY ASSURANCE

Before being released for consumption, all **Bioclin** reagents are tested by the Department of Quality Control. The quality of reagents is assured until expiration date stated on the presentation packaging, when stored and transported under appropriate conditions.

 QUIBASA QUÍMICA BÁSICA Ltda

Rua Teles de Menezes, 92 - Santa Branca
CEP 31565-130 - Belo Horizonte - MG - Brasil
Phone: +55 31 3439.5454 | E-mail: bioclin@bioclin.com.br
CNPJ: 19.400.787/0001-07 - Made in Brazil

CUSTOMER SERVICE

English

Customer Advisory Service
Phone.: 0800 0315454 | E-mail: sac@bioclin.com.br

Product exempt from ANVISA registration, in accordance with current legislation.

Review: April/2025

UNIVERSAL SYMBOLOGY

	CATALOG NUMBER		MADE BY
	LOT NUMBER		CONTROL
	MANUFACTURING DATE		POSITIVE CONTROL
	VALIDITY DATE (last day of the month)		NEGATIVE CONTROL
	TEMPERATURE LIMIT (store)		BIOLOGICAL RISK
	CONTENT IS SUFFICIENT FOR <N> TEST		FLAMMABLE
	SEE INSTRUCTIONS FOR USE		CORROSIVE
	IN VITRO DIAGNOSTIC PRODUCT		TOXIC
	KEEP AWAY FROM SUNLIGHT		DO NOT USE IF PACKAGE IS DAMAGED
	DO NOT REUSE		PRODUCT STERILIZED
	CAUTION		DANGER