Catálogo: K031-1

PROTEINAS TOTAIS MONOREAGENTE

Nº de Testes: 1250

Versão: 08/08/2018

Cód.: 00

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K031-1

Reagente Nº 1 – Biureto – 1 x 250 mL

Reagente Nº 2 – Padrão – 1 x 2 mL

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TEST CODE	0 000	SAMPLE BLANK ABSORBANCE CHECKS	
Name	PROT TOTAIS	Absorbance Limit	-0,1 3
Abbreviation	PROT	Reagent Abs Limit	-0,1 3
Mode	Ponto de Desfecho	Substrate Depletion	0 Abs
Reagent Onboard Stability	1 day	Result Type	Quantitative
Reagent Low Vol Warming	5 Tests	Primary Units	g/dL
Service Assay		Unit Conversion Factor A	-
Suppress Test Result		Unit Conversion Factor B	-
Serum	✓	Secondary Units	-
Urine	-	Decimals	1
WholeBlood (disabled)	-		
CSF (disabled)	-	REFERENCE RANGES	
(disabled)	-	Serum	
Replicates	1	Panic	0 0
Sample Blank	On Request	Male	6.0 8.0
Sample Volume	4.0 µL	Female	6.0 8.0
Reagent Volume -2.25 min (R1)	200 µL	Pediatric	0 0
Reagent Volume 2.90 min (R2)	00 µL	Urine	- -
Reagent Volume 4.70 min (R3)	00 µL	Whole Blood	- -
WAVELENGTH	546 nm	CALIBRATION TYPE	Calibrator
Incubation Time	4.5 min	Correlation	Factor Offset
ABSORBANCE CHECKS		Calibrator Name	Biocal – K072
Absorbance Limit (Abs)	-0,1 3	Replicates	1
Reagent Abs Limit (Abs)	-0,1 3	Expiration Time	7 days
Reaction Direction	Increasing	Auto Predilution	-
Reagent Deviation Abs	-	Reagent Blank	✓
Substrate Depletion	0 Abs	Reag. Blank Expiration	1 day
Alinearity Limit	10 %	Reag. Blank Replicates	1
		Reag. Blank Abs Limits	-0,1 3
		Number of Points	1
		Algorithm	Linear
		Algorithm Inaccuracy	5%
		Extended Abs Limits	1 1
		Extended Concent Limits	0 10

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