Catálogo: K056-1

URÉIA UV

Nº de Testes: 665

Versão: 08/08/2018

Cód.: 00

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K056-1

Reagente Nº 1 – Tampão – 2 x 80 mL

Reagente Nº 2 – Reagente Enzimático – 1 x 40 mL

Reagente Nº 3 – Padrão – 1 x 3 mL

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TEST CODE	0 000	SAMPLE BLANK ABSORBANCE CHECKS
Name	URÉIA UV	Absorbance Limit -0,1 3
Abbreviation	URE	Reagent Abs Limit -0,1 3
Mode	Two Point	Substrate Depletion 0 Abs
Reagent Onboard Stability	1 day	Result Type Quantitative
Reagent Low Vol Warming	5 Tests	Primary Units mg/dL
Service Assay		Unit Conversion Factor A -
Suppress Test Result		Unit Conversion Factor B -
Serum	✓	Secondary Units -
Urine	-	Decimals 1
WholeBlood (disabled)	-	REFERENCE RANGES
CSF (disabled)	-	Serum
(disabled)	-	Panic 0 0
Replicates	1	Male 15 40
Sample Blank	On Request	Female 15 40
Sample Volume	2.0 µL	Pediatric 0 0
Reagent Volume -2.25 min (R1)	200 µL	Urine - -
Reagent Volume 2.90 min (R2)	00 µL	Whole Blood - -
Reagent Volume 4.70 min (R3)	00 µL	CALIBRATION TYPE
WAVELENGTH	340 nm	Correlation
Two Point	Point 1 24 sec	Factor 1.000
	Point 2 103 sec	Offset 0.000
ABSORBANCE CHECKS		Calibrator Name Biocal – K072
Absorbance Limit (Abs)	-0,1 3	Replicates 1
Reagent Abs Limit (Abs)	-0,1 3	Expiration Time 7 days
Reaction Direction	Decreasing	Auto Predilution -
Reagent Deviation Abs	-	Reagent Blank ✓
Substrate Depletion	0 Abs	Reag. Blank Expiration 1 day
Alinearity Limit	10 %	Reag. Blank Replicates 1
		Reag. Blank Abs Limits -0,1 3
		Number of Points 1
		Algorithm Linear
		Algorithm Inaccuracy 5%
		Extended Abs Limits 1 1
		Extended Concent Limits 0 10

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